



created with love by



Changes and New features (Avaz ver 6.6+)

For upgrading users (iOS and Android)

-
-
- 1. Changes to the app**
 - 2. New features added**
 - 3. Features removed**
-
-


Changes to the app

— Before - After —


Subscription status and Purchase Now button



Before

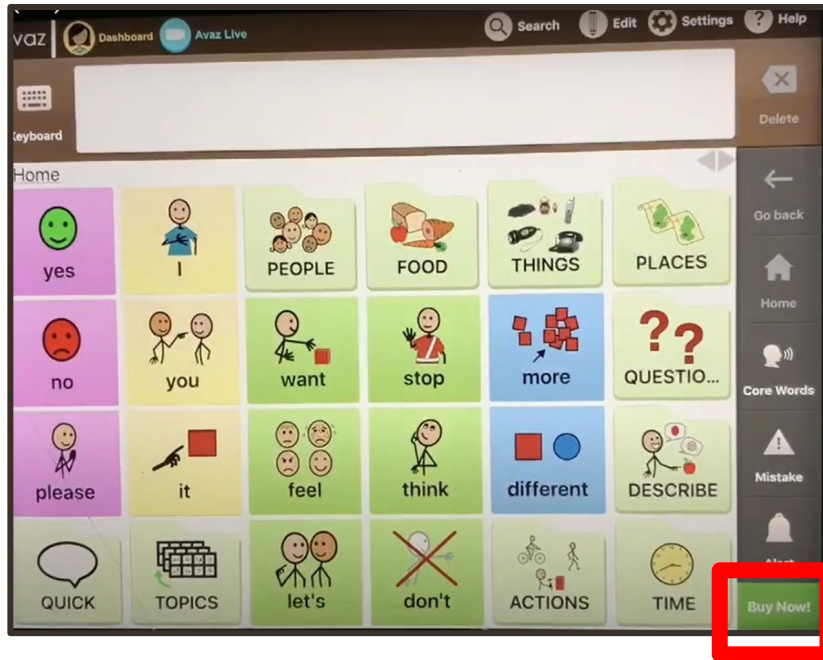

After

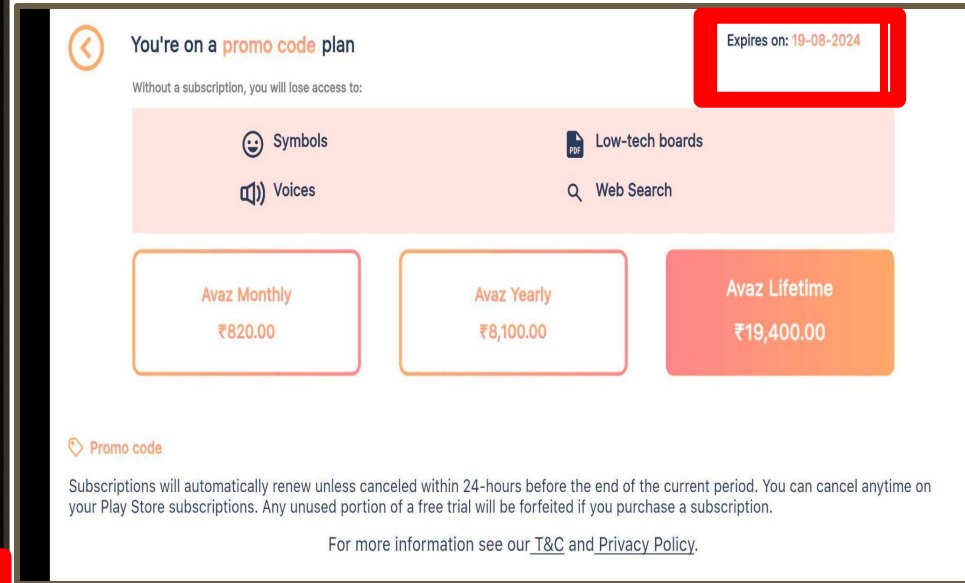

App **Subscription / Expiry** status was visible on the top left of home screen & **Purchase Now** button on bottom right corner.


Now, App subscription / expiry status is not visible on Home Page.

Menu > Settings > Account details


Note: Lifetime subscription will not have Account details at all.

In-App Subscription

Before

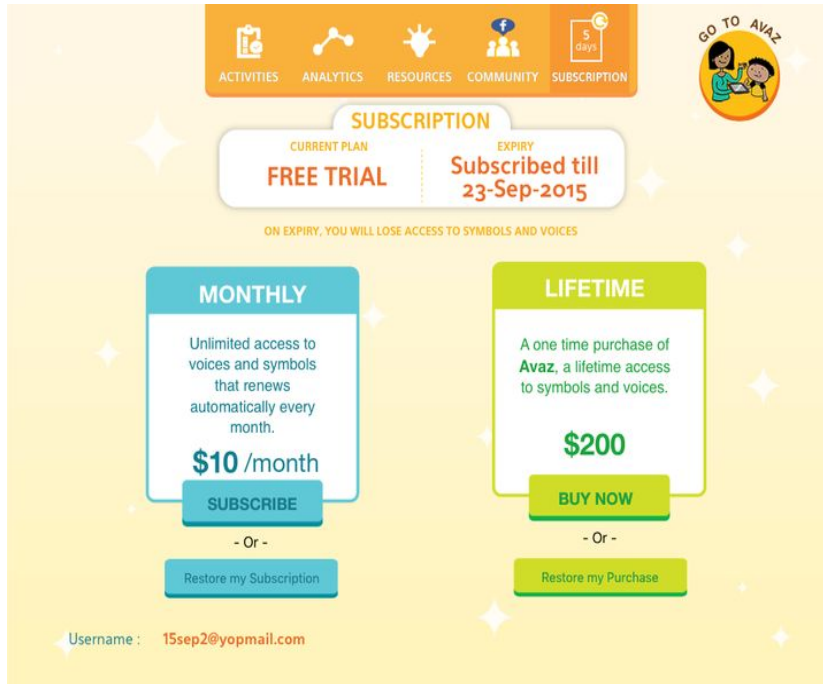

After

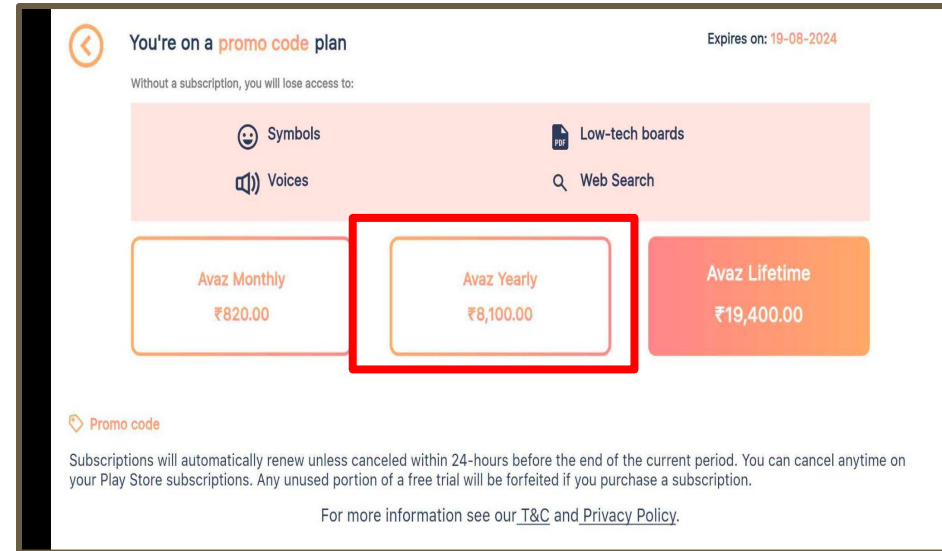


Settings > Subscribe to Symbols & Voices > Monthly / Annual / Lifetime

Menu > Settings > Account details

Note: Lifetime subscription will not have Account details at all.


Purchase Now button

Before

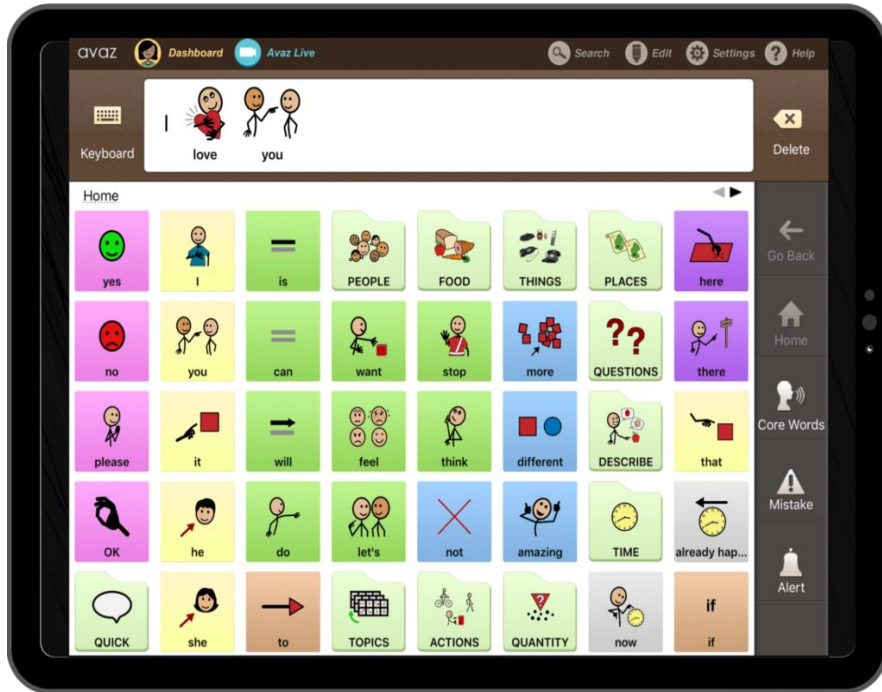

App **Subscription / Expiry** status was visible on the top left of home screen & **Purchase Now** button on bottom right corner.

After

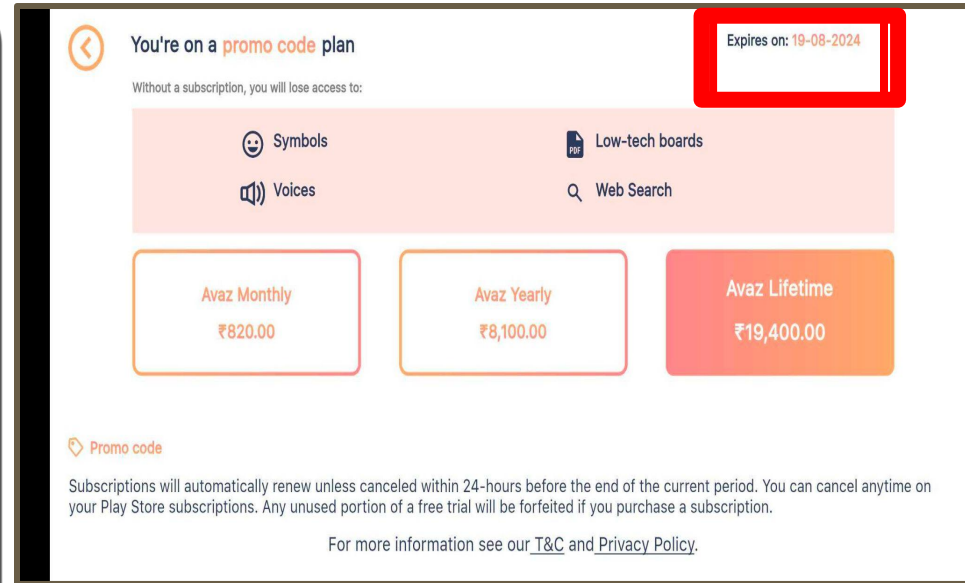


Now, App subscription / expiry status is not visible on Home Page.

Menu > Settings > Account details

Note: Lifetime subscription will not have Account details at all.

Search, Edit, Settings

Before

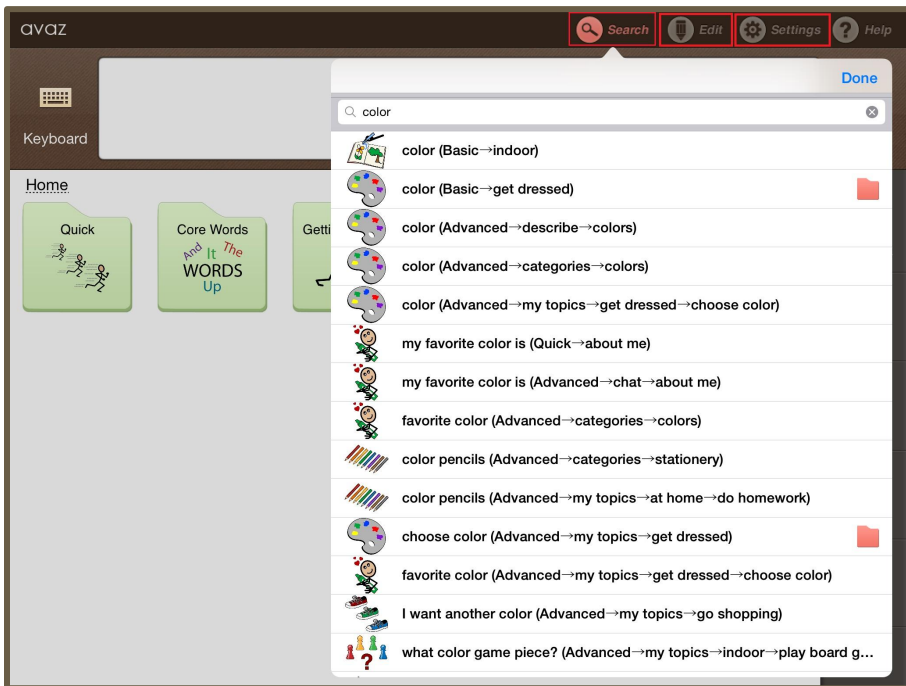

After




Search, Edit & Settings were available on the top bar

Tap on Menu


1. Edit Words
2. Search
3. Settings

How to customize - add new word / folder

Before

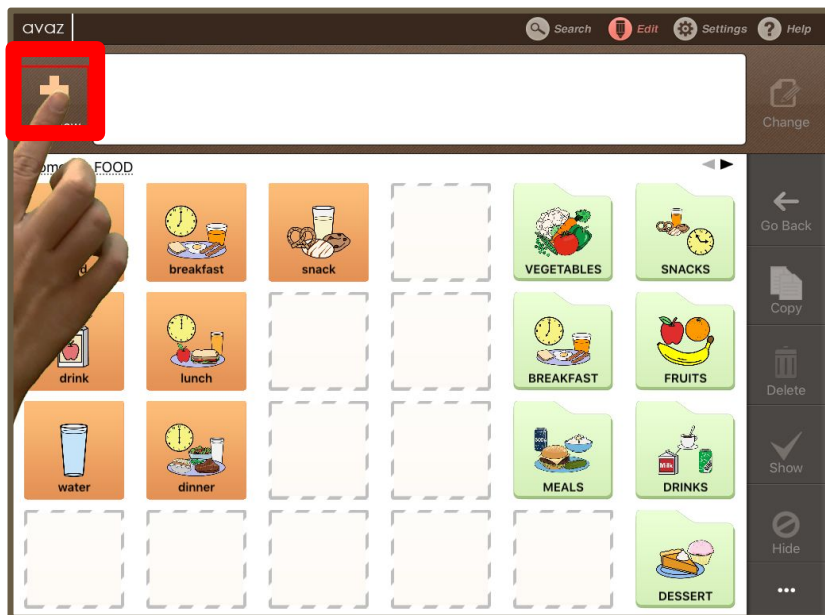

After

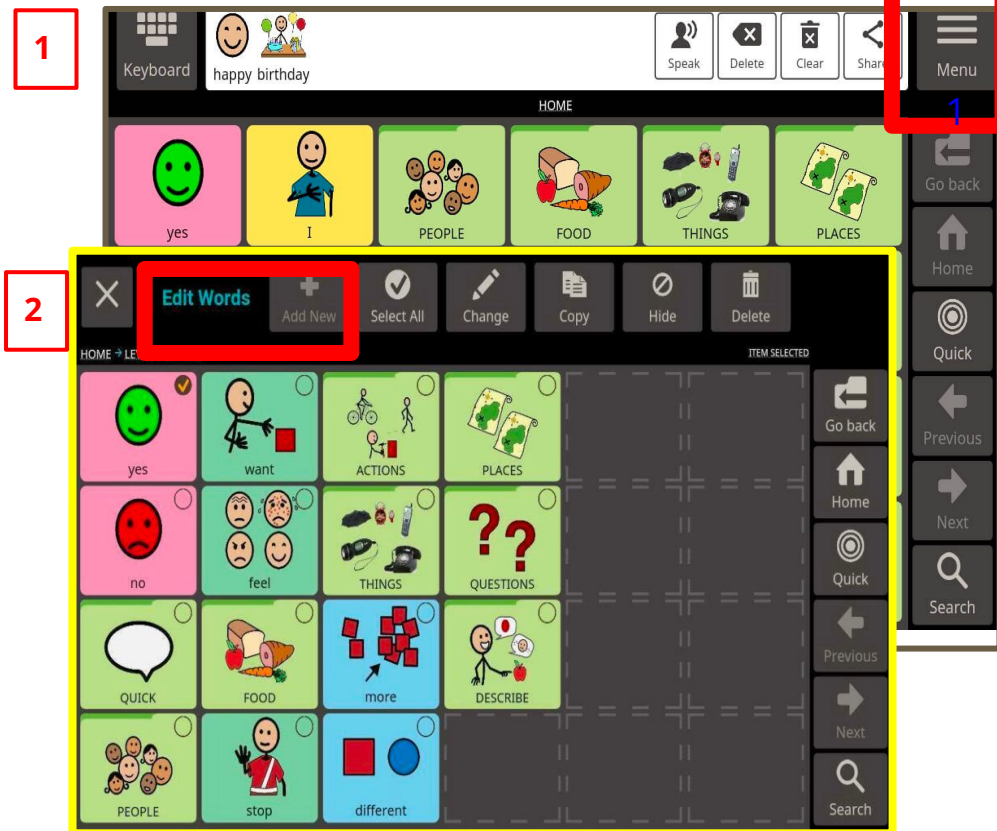


Edit > Add New > New Message / New Category

Menu > Edit Words > Add New


How to customize - Take Photos

Before

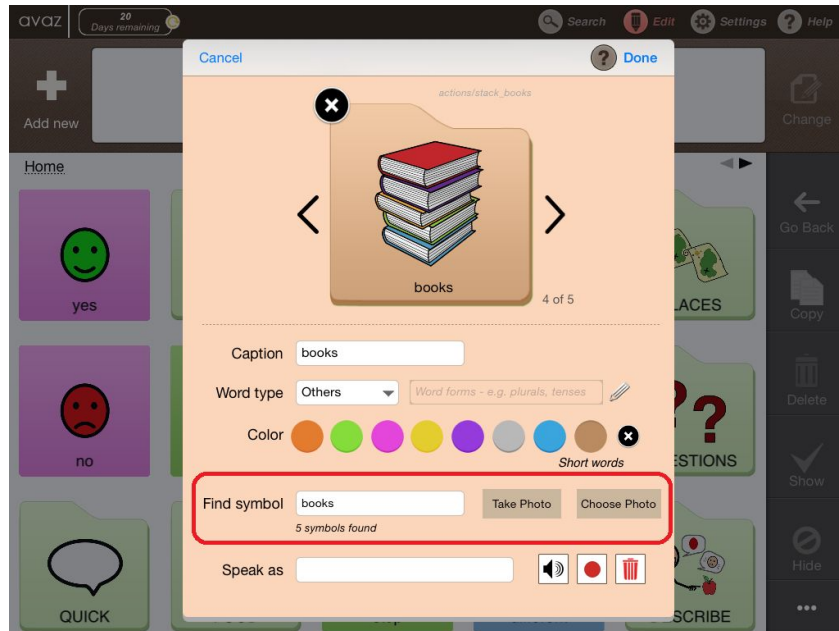

Edit > Add New > New Message / New Category


After

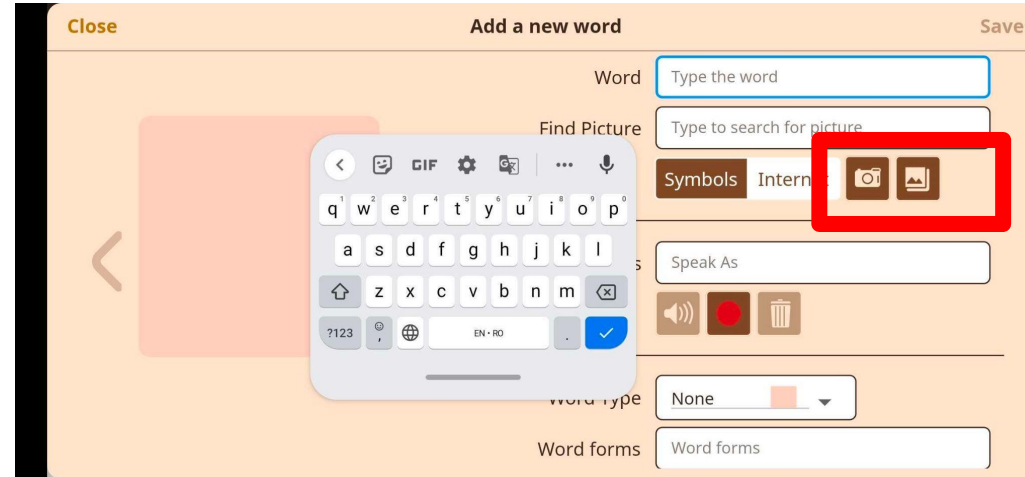

Menu > Edit Words > Add a new word


How to add Photos - Internet

Before

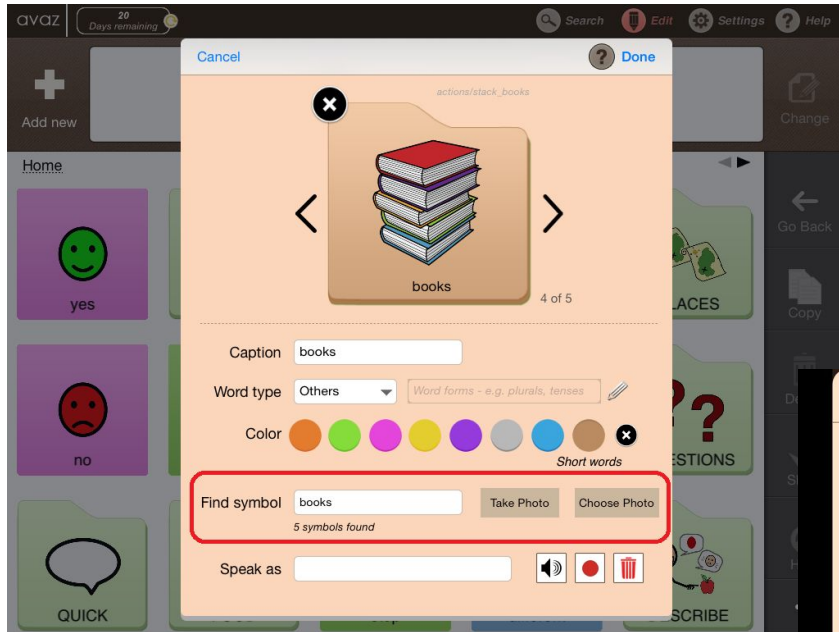

Edit > Add New > New Message / New Category


After

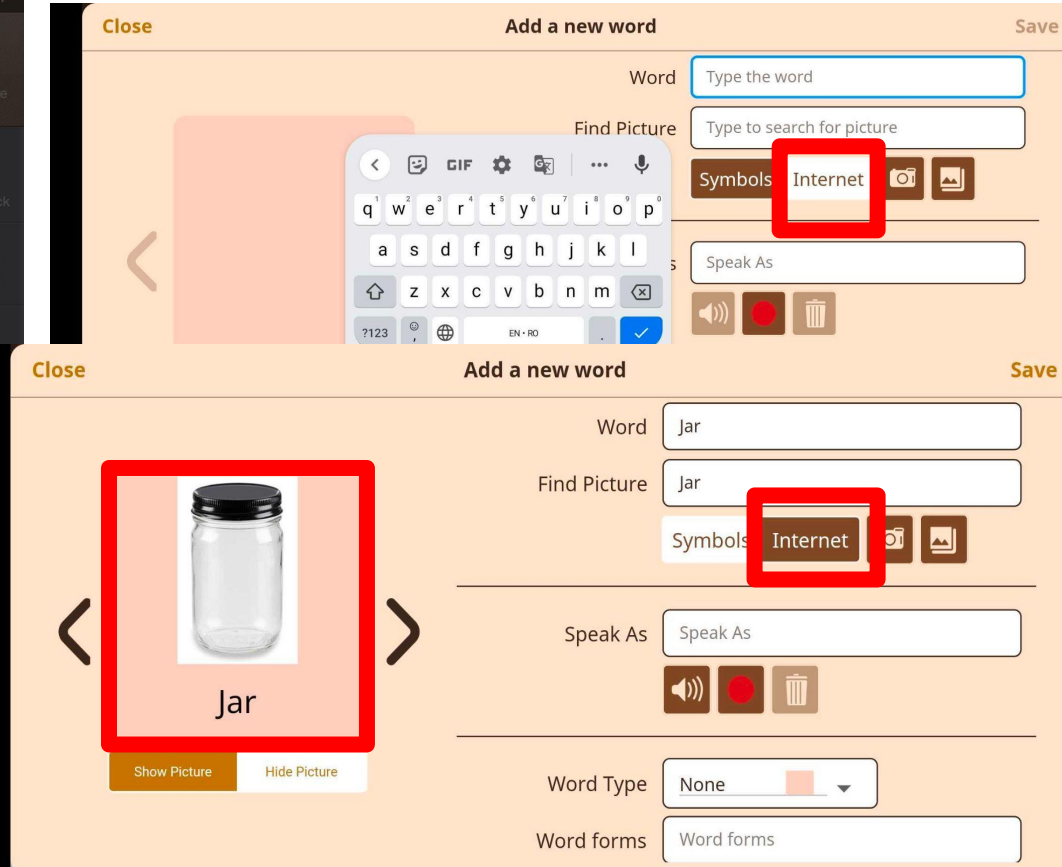

Menu > Edit Words > Add a new word


Customizing words : Copy / Paste / Hide / Show / Change / Delete

Before

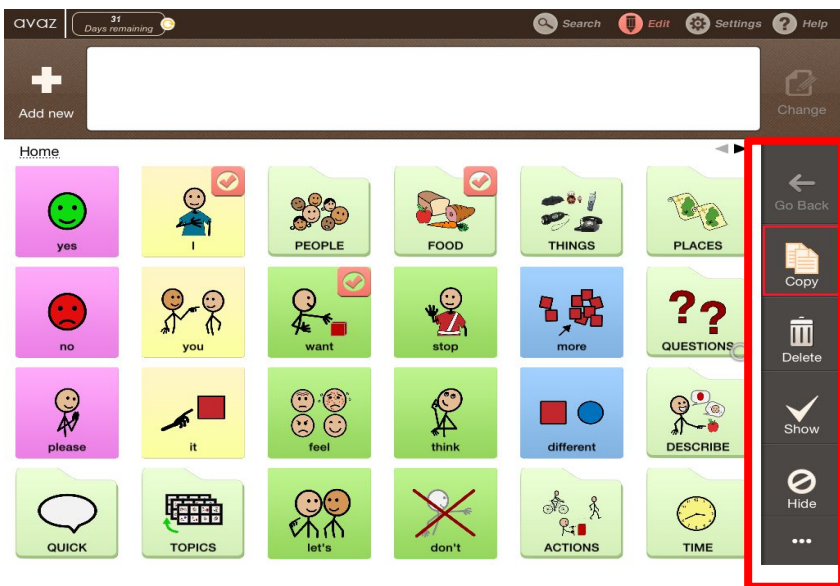

Edit > Change / Copy / Paste / Hide / Show / Delete


After

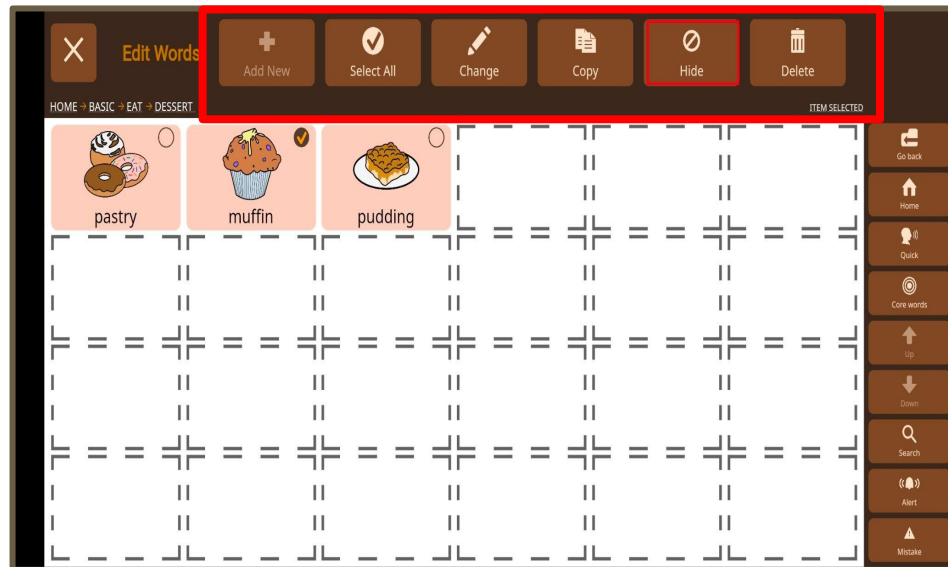

Menu > **Edit Words** > Change / Copy / Paste / Hide / Show / Delete

High Contrast (before) = Dark Themes (after)

Before

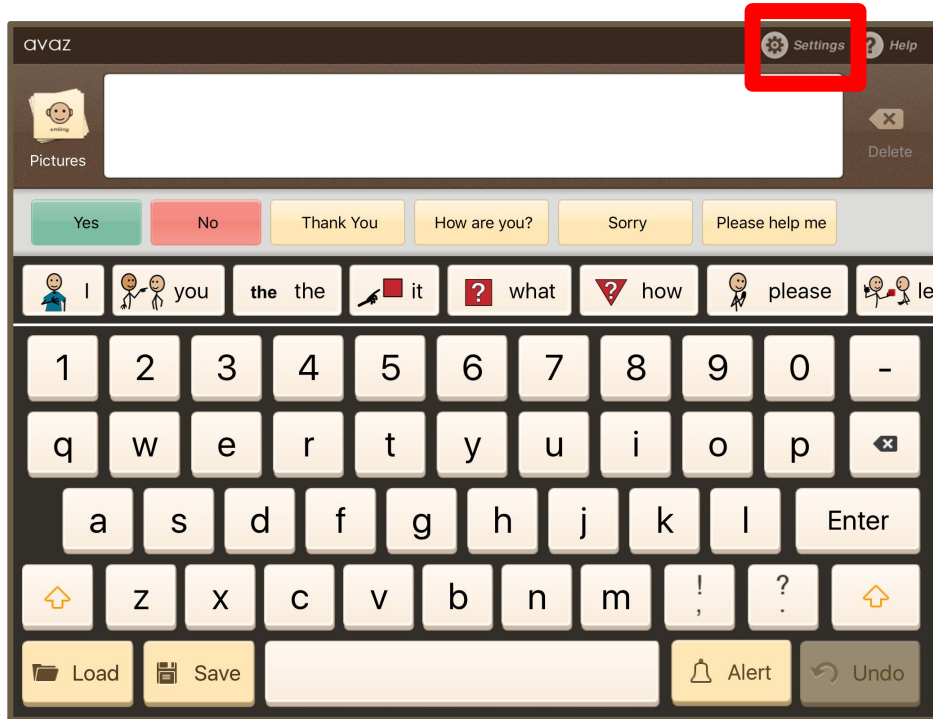

Settings > High Contrast = ON


After



Menu > Settings > Theme > Light (Classic)/Dark


Save frequently used phrases in Keyboard

Before

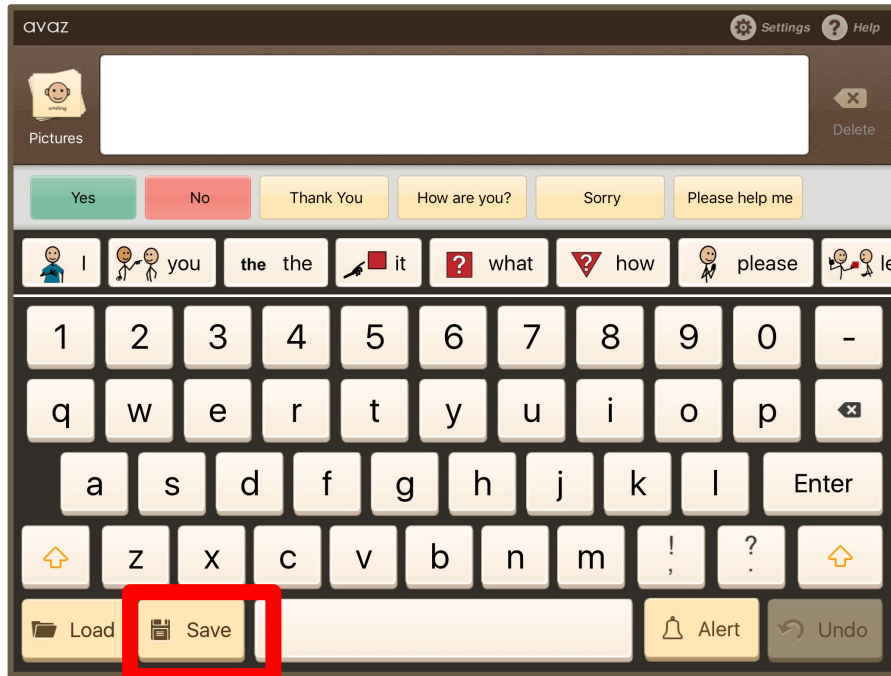

1. Tap **Save** button.


After

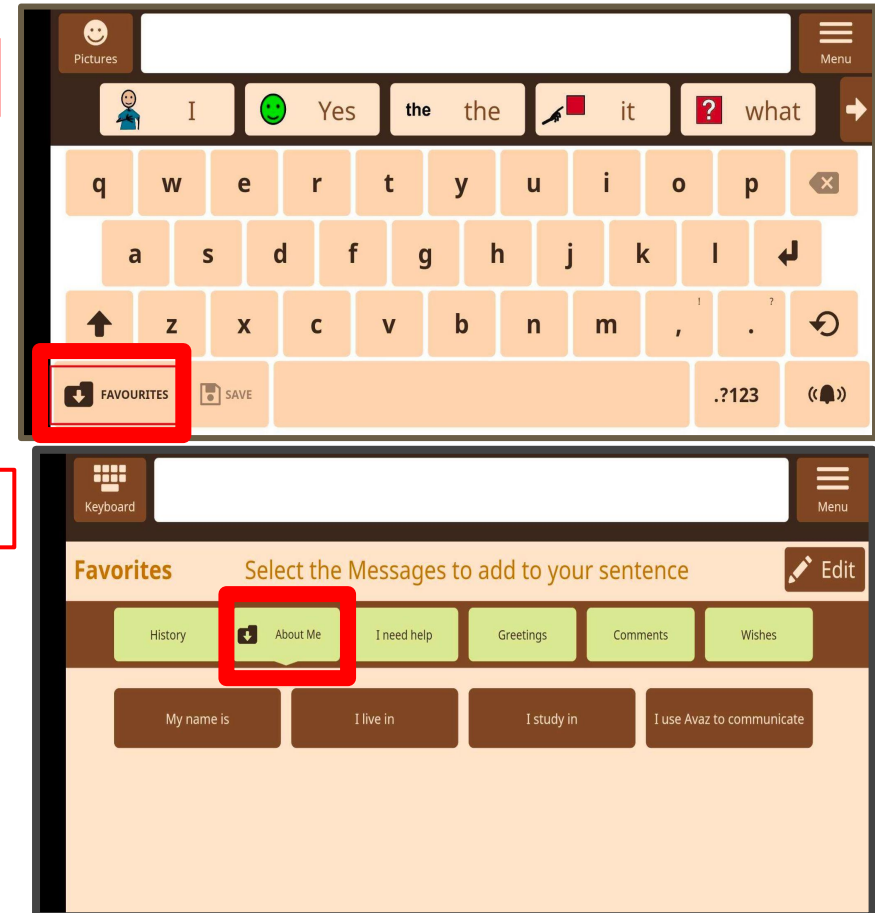

1. Tap **Favourites** button.
2. Select the folder to save it in.


Create Low Tech / Avaz Book as PDF

Before

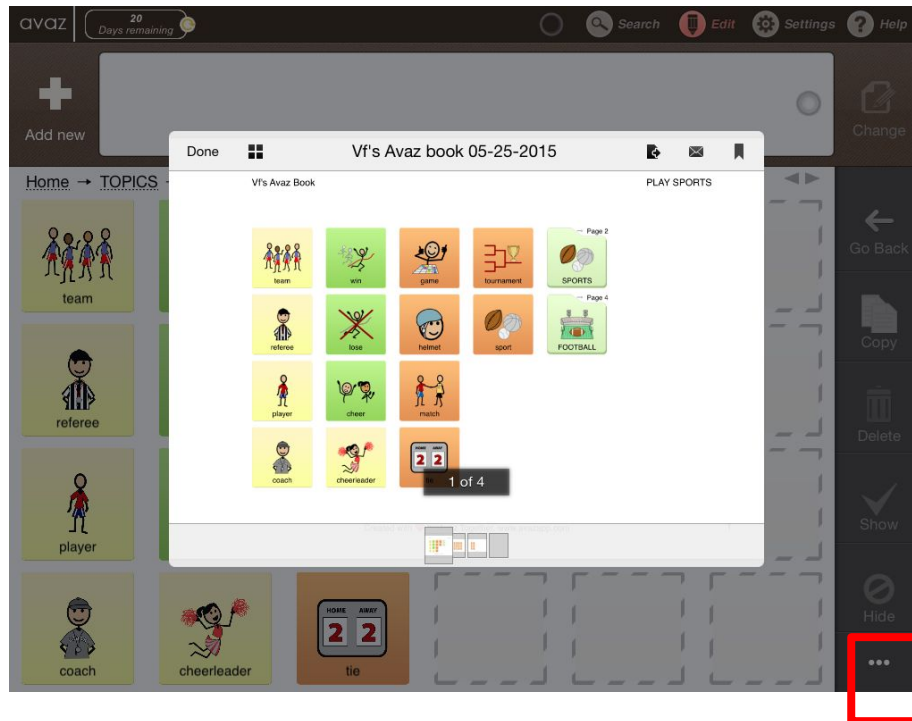

Edit > .. > Create Avaz book

After

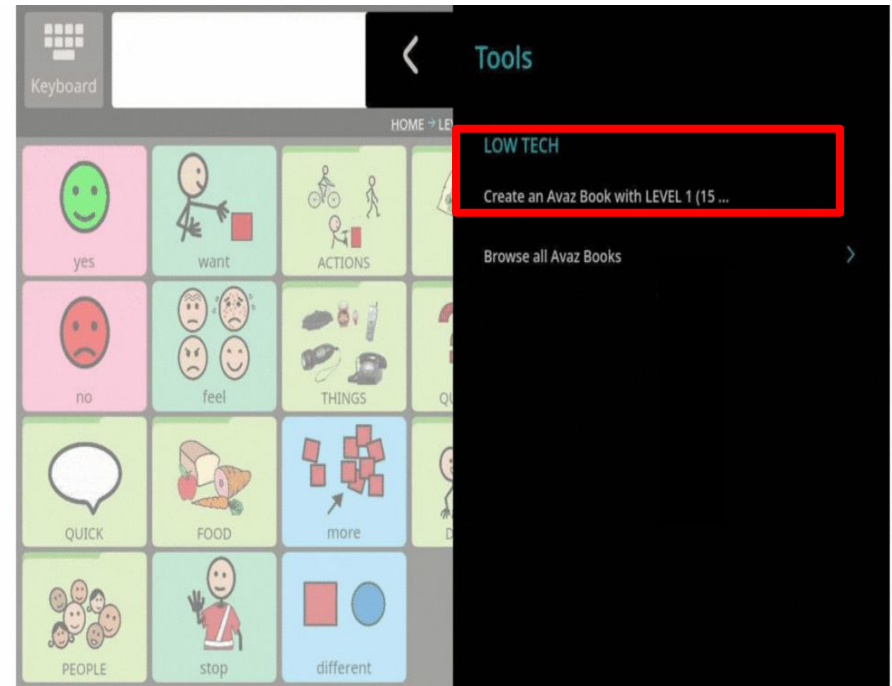

Menu > Tools > Low Tech > Create an Avaz book

Word forms (morphology) display

Before



Settings > Grammar = ON

Word forms are displayed horizontally below the message box

After




Menu > Settings > Grammar > Grammar = ON

If there are variations of a word, you will see a small "+" symbol on the top right corner of the image. Click it to see the various word forms displayed.

Export/Import




Before

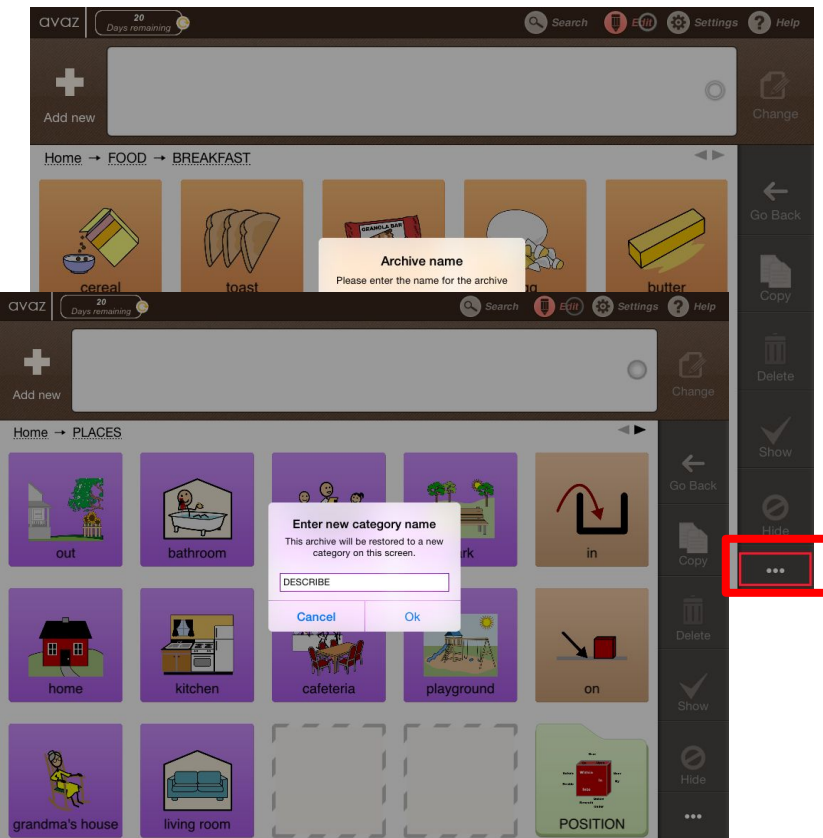

Edit > ... (right bottom corner) > Export / Import

Exported folders are available under **Settings** > **Backup & Restore**.

After

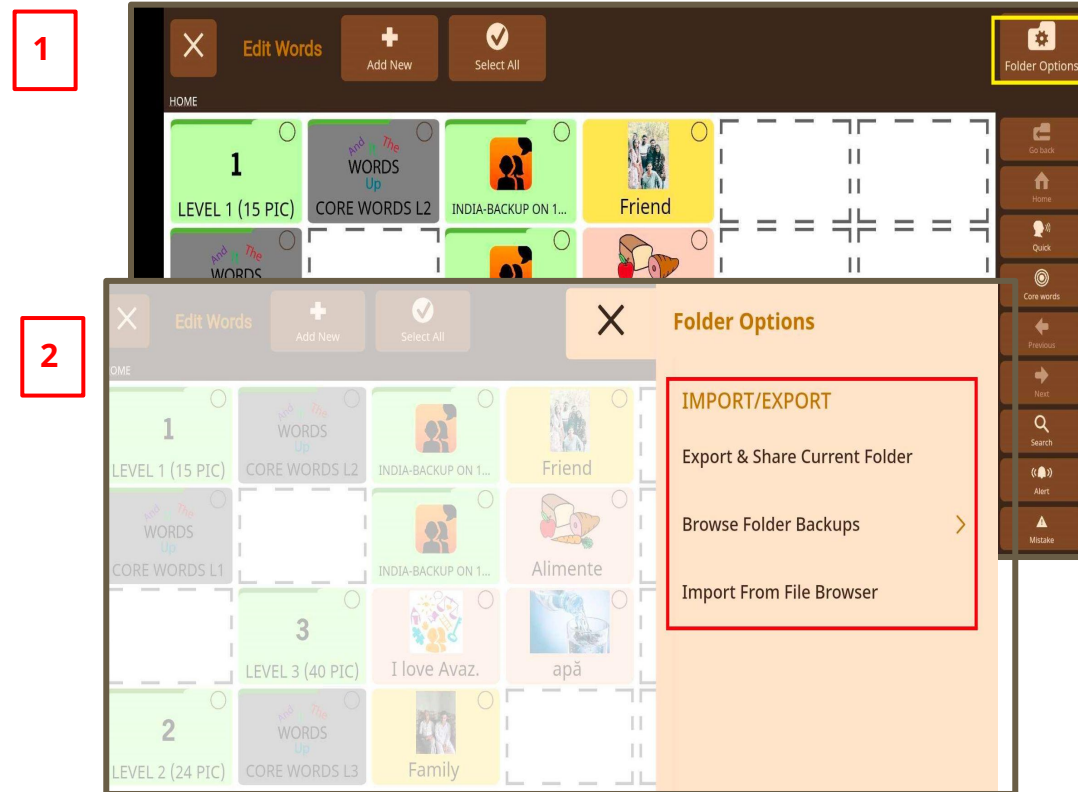


Menu > Edit words > Folder Options > Export / Import

Exported folders are available under **Folder Options** > **Browse Folder Backups**.




Enlarge word on selection - Zoom

Before

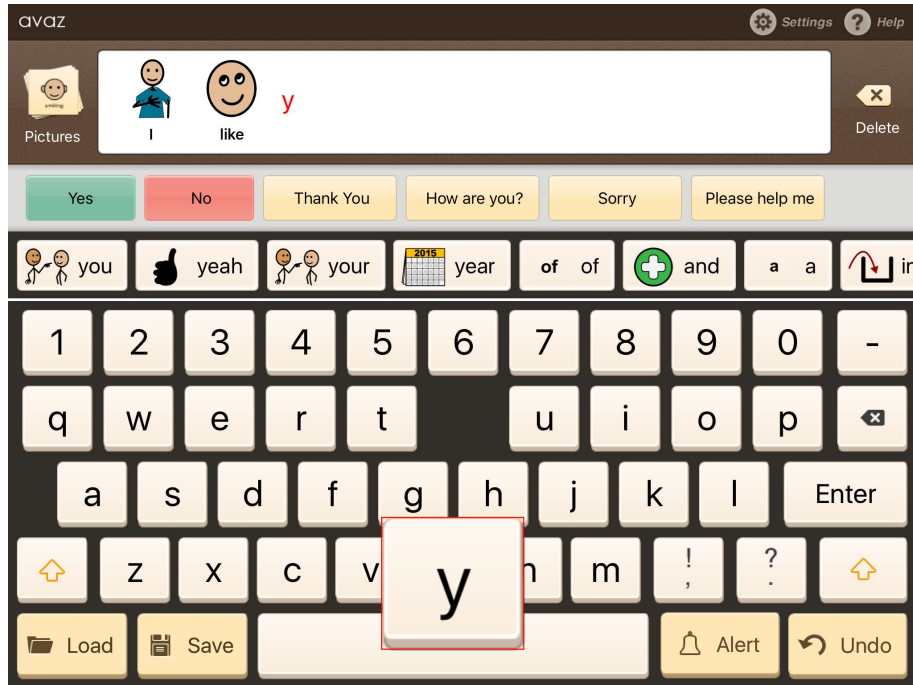

Tap **Settings > Zoom on select**(under **Picture settings**) to configure zoom options. You can set it to zoom at slow speed, or fast speed or no zoom at all. The default is set at medium speed.

After

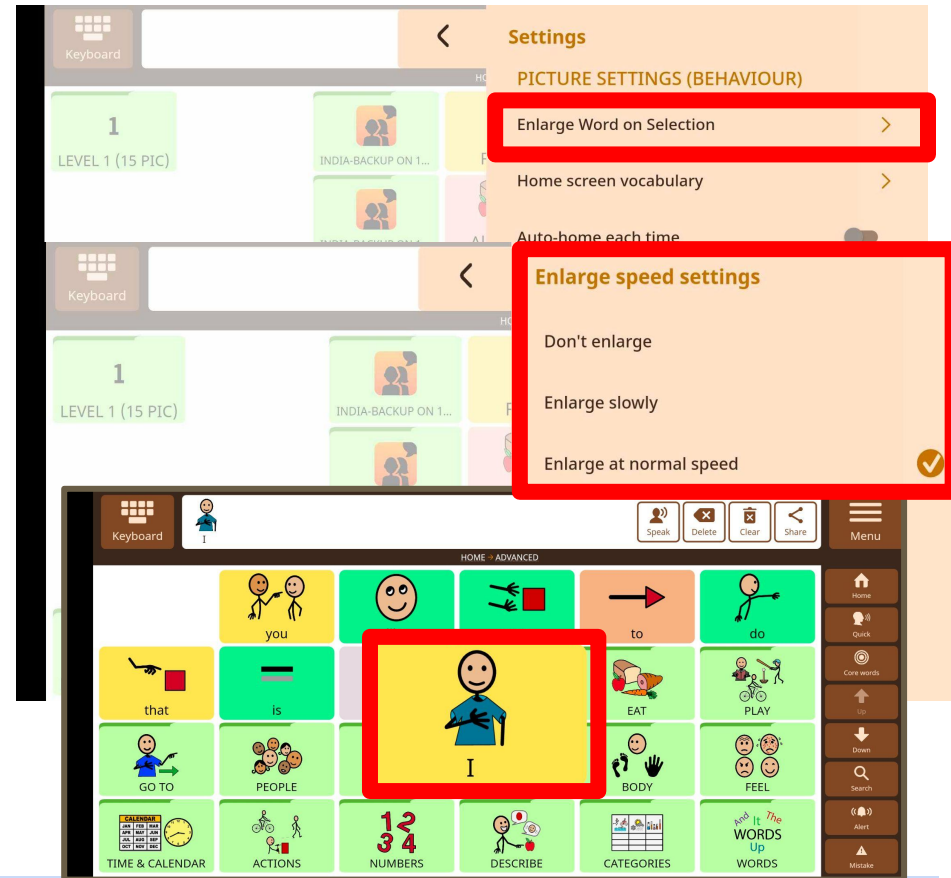


Menu > Settings > Enlarge word on selection> Enlarge speed settings

Options: Don't enlarge, Enlarge slowly, Enlarge at Normal Speed & Enlarge Quickly


Avaz Support



Before

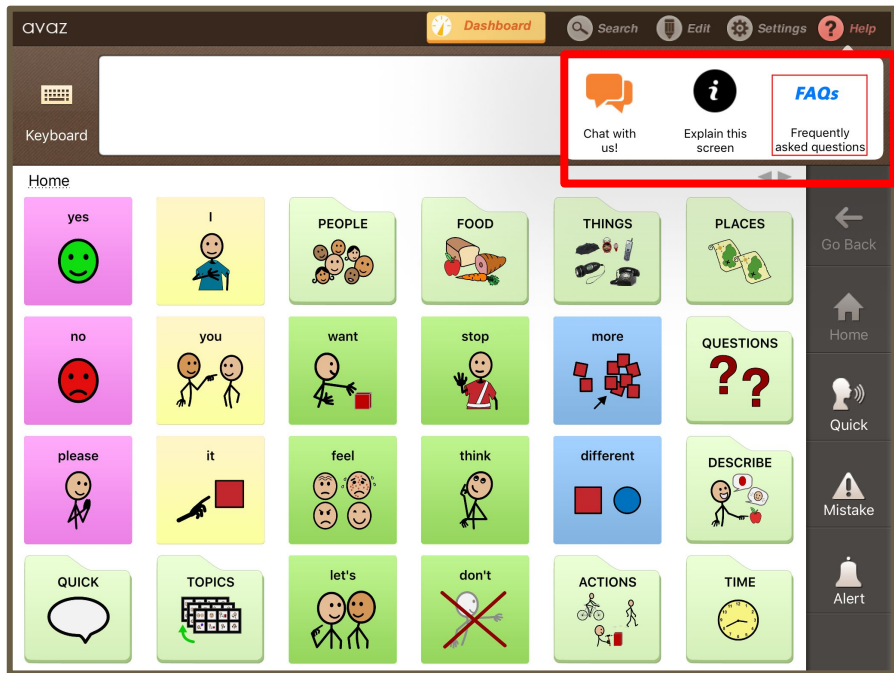

After

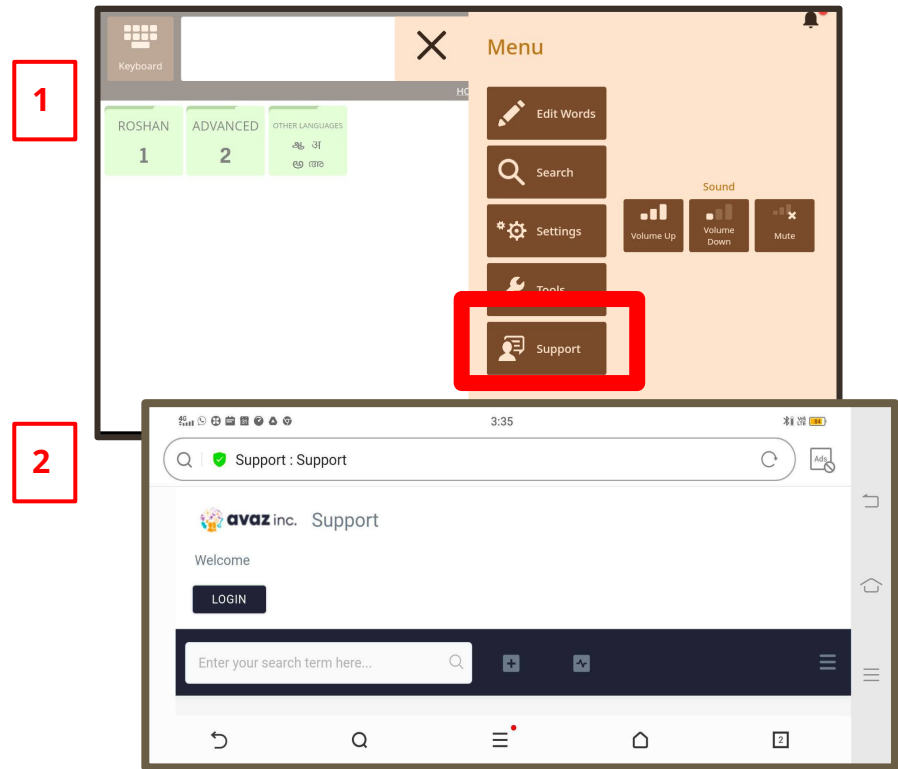

Help > FAQ / Chat / Explain this screen(Screenshots)

Menu > Support > Contact Us (email) / Get support

Get Support opens the Avaz Support FAQ page on a new tab on your browser- to browse FAQs and also raise tickets.

Note: *Chat* and *Explain Screen* are discontinued.



Newly added features

— **Avaz ver 6.6+** —

Speak, Clear and Share from Message box



Explicit Speak, Clear and Share buttons are added in the Message box.

Menu button added on Home page



New Menu button added on Home page, with Settings, Edit, Support going under it.

Select All/Unselect All



Menu > Edit Words > Select All / Unselect all

Select All/Unselect All



Edit Words

Done Add New **Unselect ...** Change Copy Hide Delete


HOME → FOOD → VEGETABLES 13 ITEMS SELECTED

carrot	cauliflo...	potato	corn	vegeta...
peas	brussel ...	onion	zucchi...	
beans	broccoli	tomato	spinach	


Go back Home Quick Core ... Previous Next Search

Menu > Edit Words > Select All / Unselect all

Search button on Side Navigation bar

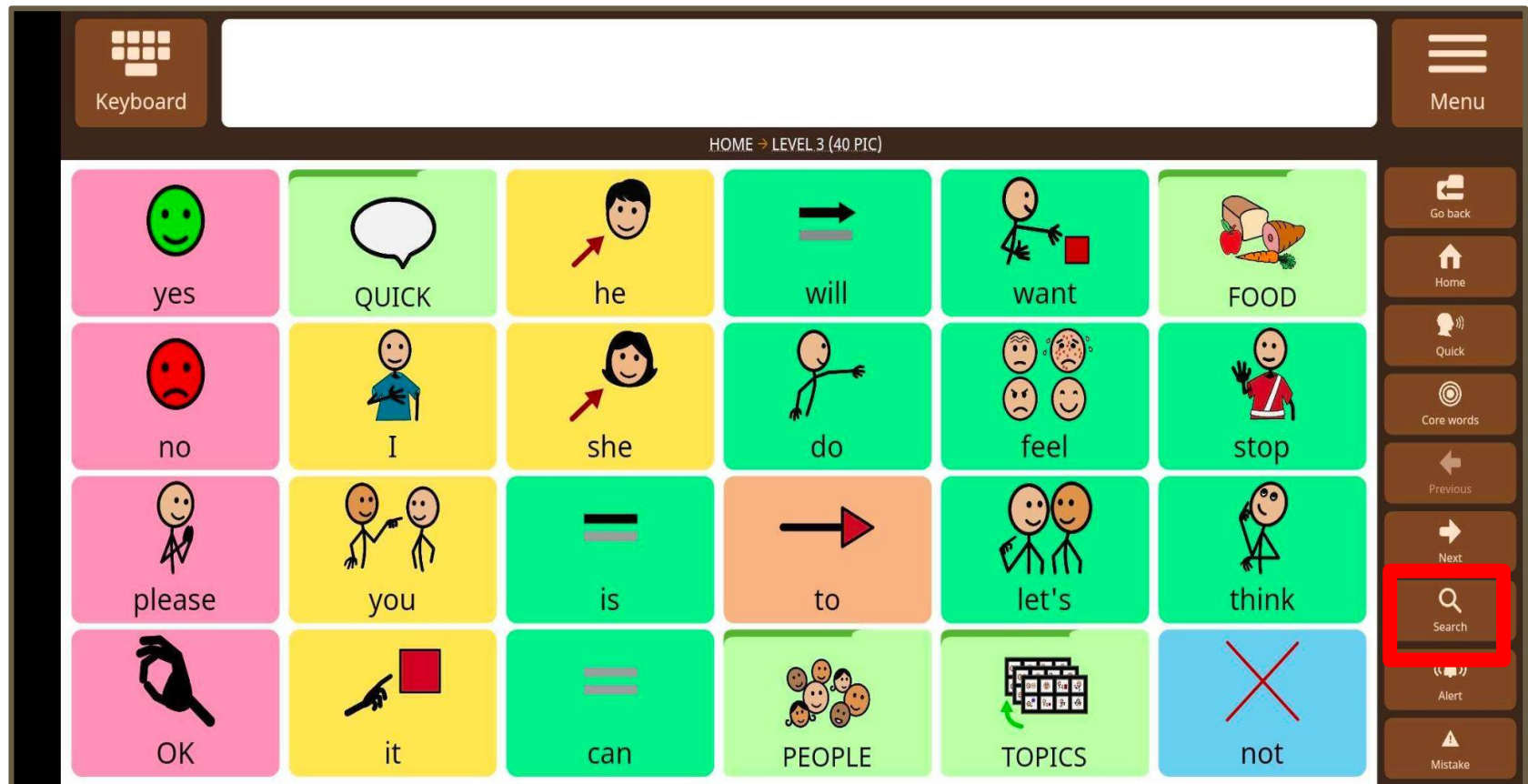


Search - See Path

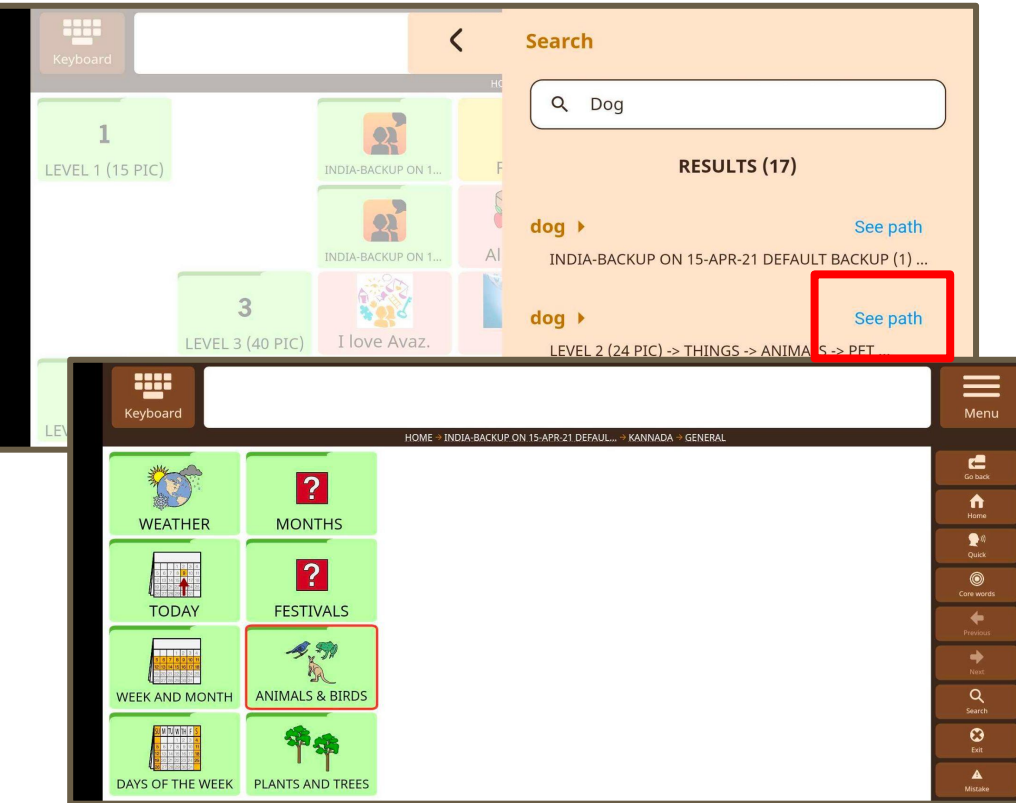

Search > Add “word” to search > Results > **See Path**

This automatically takes you through the path to reach the word, with a visual hint for each word navigated.

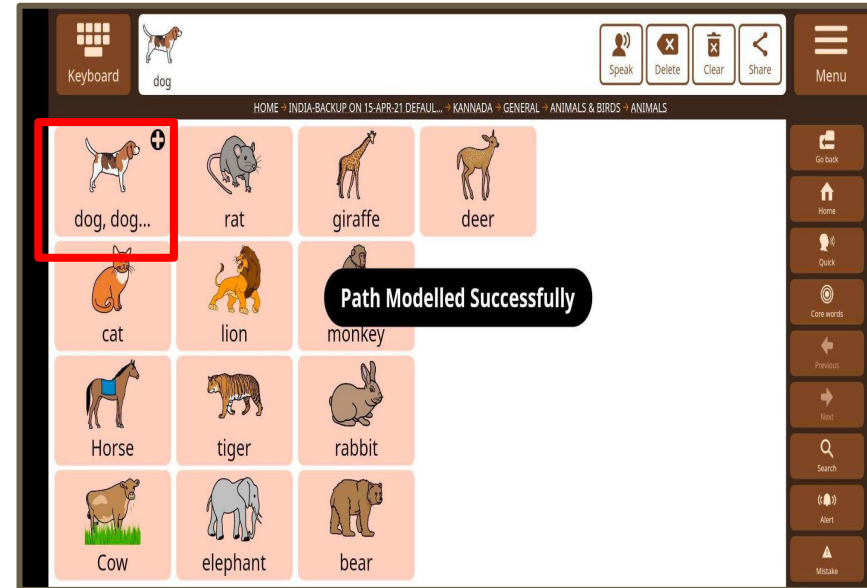


Menu > Search > Add “word”> Results > **See path**

See Path needs a manual control now. This shows the path to reach the word, but each word has to be manually tapped and in the end it says “Path Modelled Successfully”.

Vocab reorganization for Grid sizes of 60 and 77

1

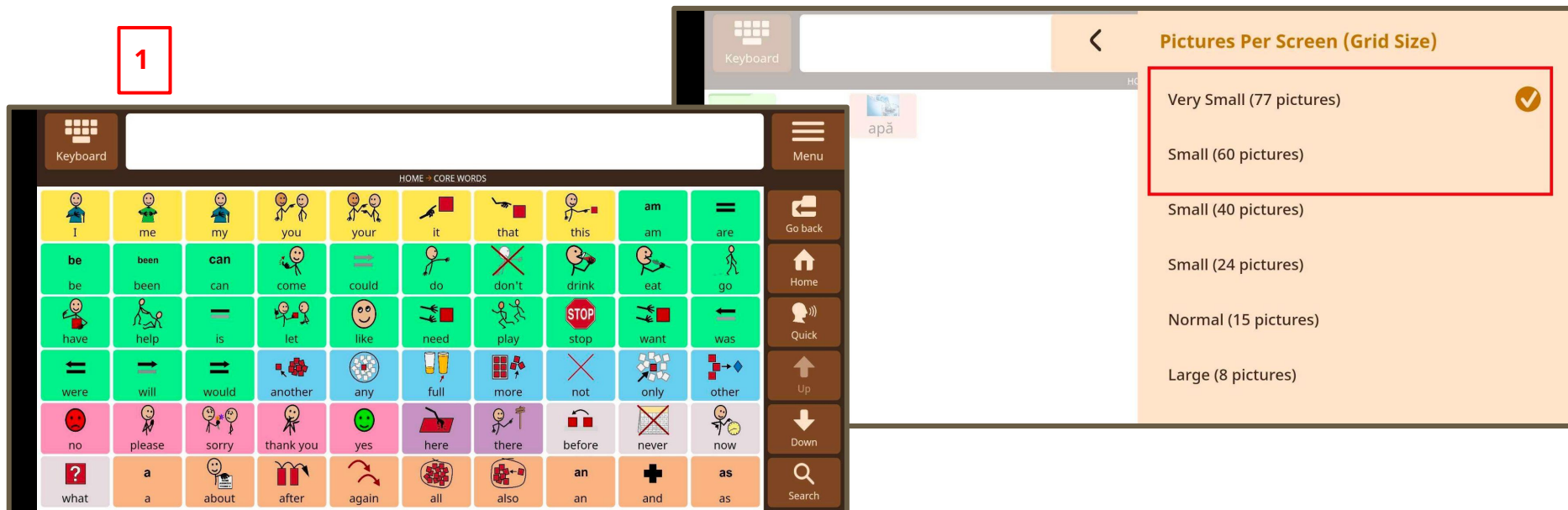

Keyboard

Menu

HOME - CORE WORDS


am am are = be been can come could do don't drink eat go have help is let like need play stop want was were will would another any full more not only other up down search

Pictures Per Screen (Grid Size)

- Very Small (77 pictures)
- Small (60 pictures)
- Small (40 pictures)
- Small (24 pictures)
- Normal (15 pictures)
- Large (8 pictures)

1. 60 Pics per screen
2. 77 pics per screen

2

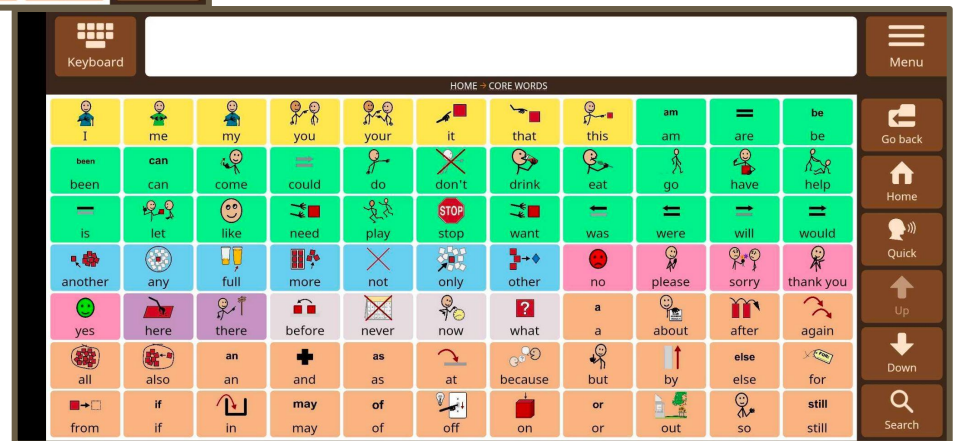

Keyboard

Menu

HOME - CORE WORDS


am am are = be been can come could do don't drink eat go have help is let like need play stop want was were will would another any full more not only other up down search

Pictures Per Screen (Grid Size)

- Very Small (77 pictures)
- Small (60 pictures)
- Small (40 pictures)
- Small (24 pictures)
- Normal (15 pictures)
- Large (8 pictures)

Menu > Settings > Picture Settings (Appearance) > Pictures per screen (Grid size)



Customizable Side Navigation bar

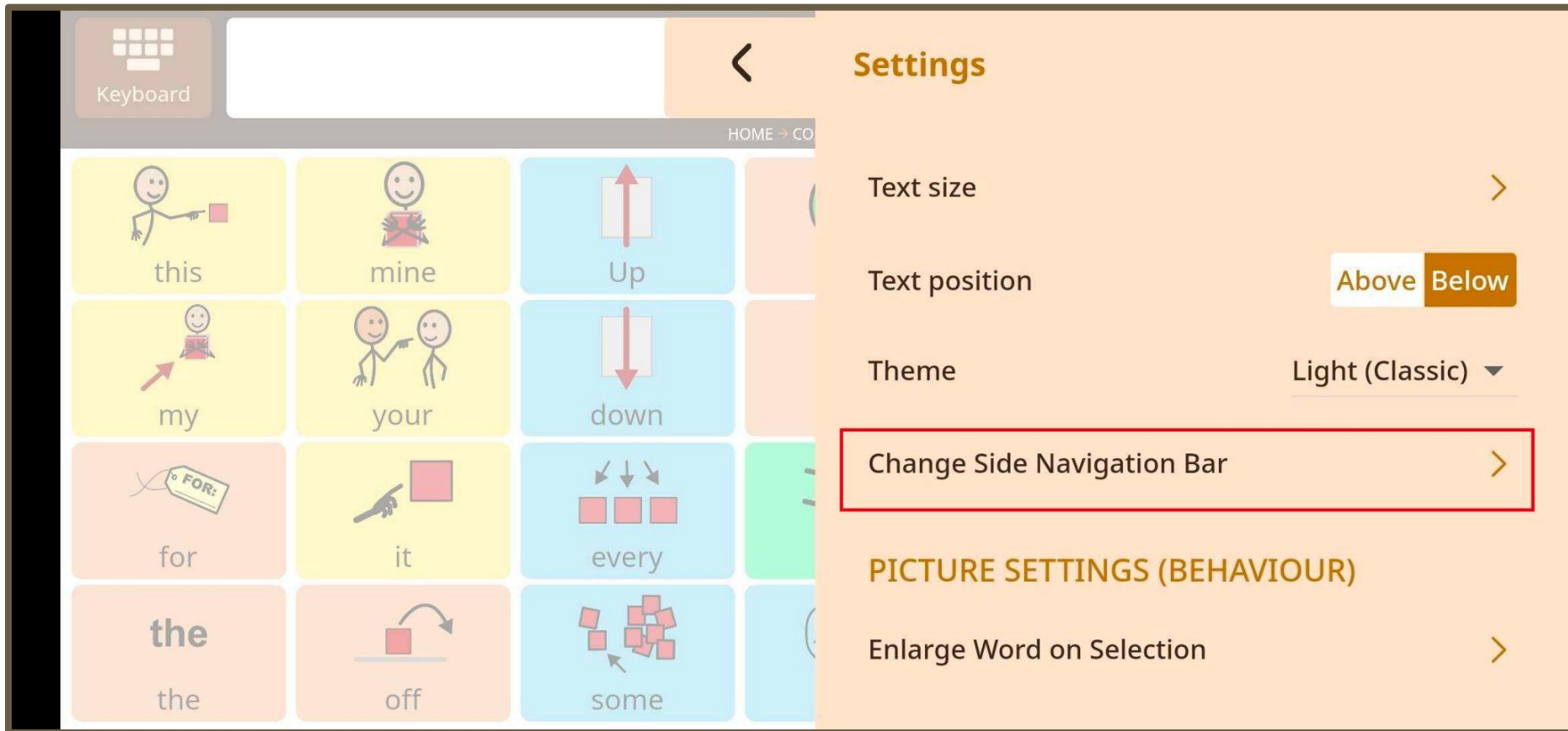

The screenshot displays the avaz AAC app interface. On the left, a grid of communication tiles is visible, including words like 'this', 'mine', 'Up', 'my', 'your', 'down', 'for', 'it', 'every', 'the', 'the', 'off', and 'some'. A 'Keyboard' icon is at the top left. On the right, the 'Settings' menu is open, showing options for 'Text size', 'Text position' (set to 'Above'), 'Theme' (set to 'Light (Classic)'), and 'Change Side Navigation Bar' (highlighted with a red box). Below these are 'PICTURE SETTINGS (BEHAVIOUR)' and 'Enlarge Word on Selection'.

Menu > Settings > Change Side Navigation Bar

Customize Side Navigation Bar : Position

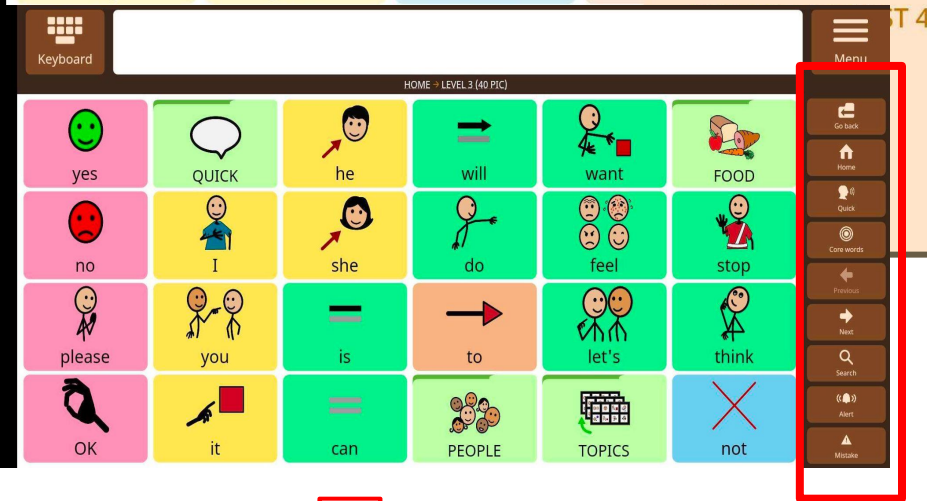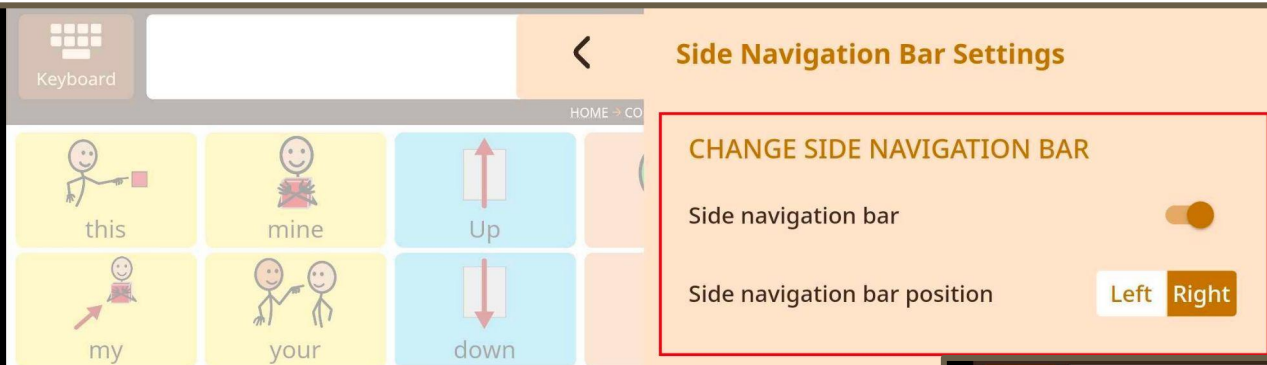

1


Placement of the side Navigation bar - on Right



2

Placement of the side navigation bar - on Left

Remove Side Navigation Bar

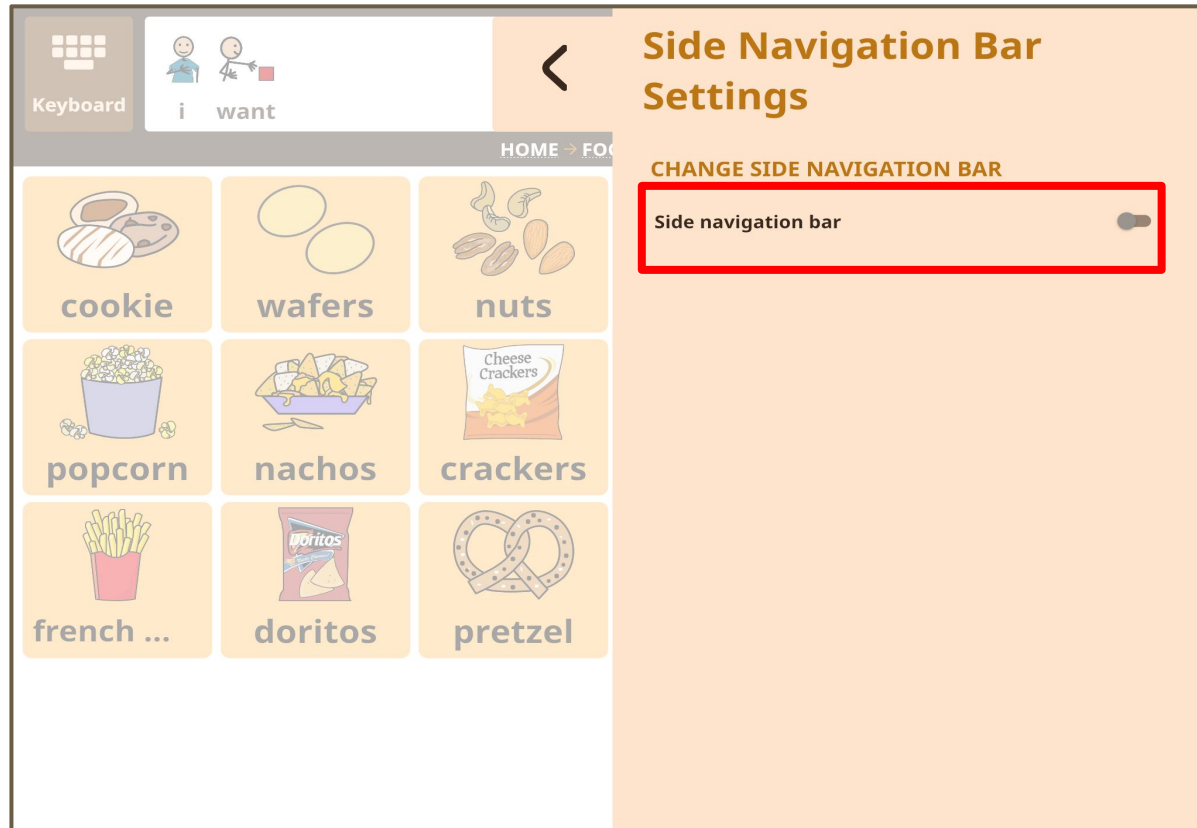



The side Navigation bar can also be removed completely.


Customize Side Navigation Bar - Buttons


Keyboard


HOME → CO



this



mine



Up



my



your



down

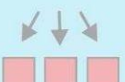

for


it


every


the


off


some

Side Navigation Bar Settings


CHOOSE ATLEAST 4 BUTTONS

- Go Back ☒
- Home ☒
- Quick ☒
- Core Words ☒
- Next Page ☒
- Previous Page ☒

Select buttons to display on Side navigation bar (Minimum 4 buttons):

- Go Back, Home, Quick / Core Words, Next Page, Previous Page, Search Words, Alert, I Made A Mistake

Word forms (Morphology) display in Grid format

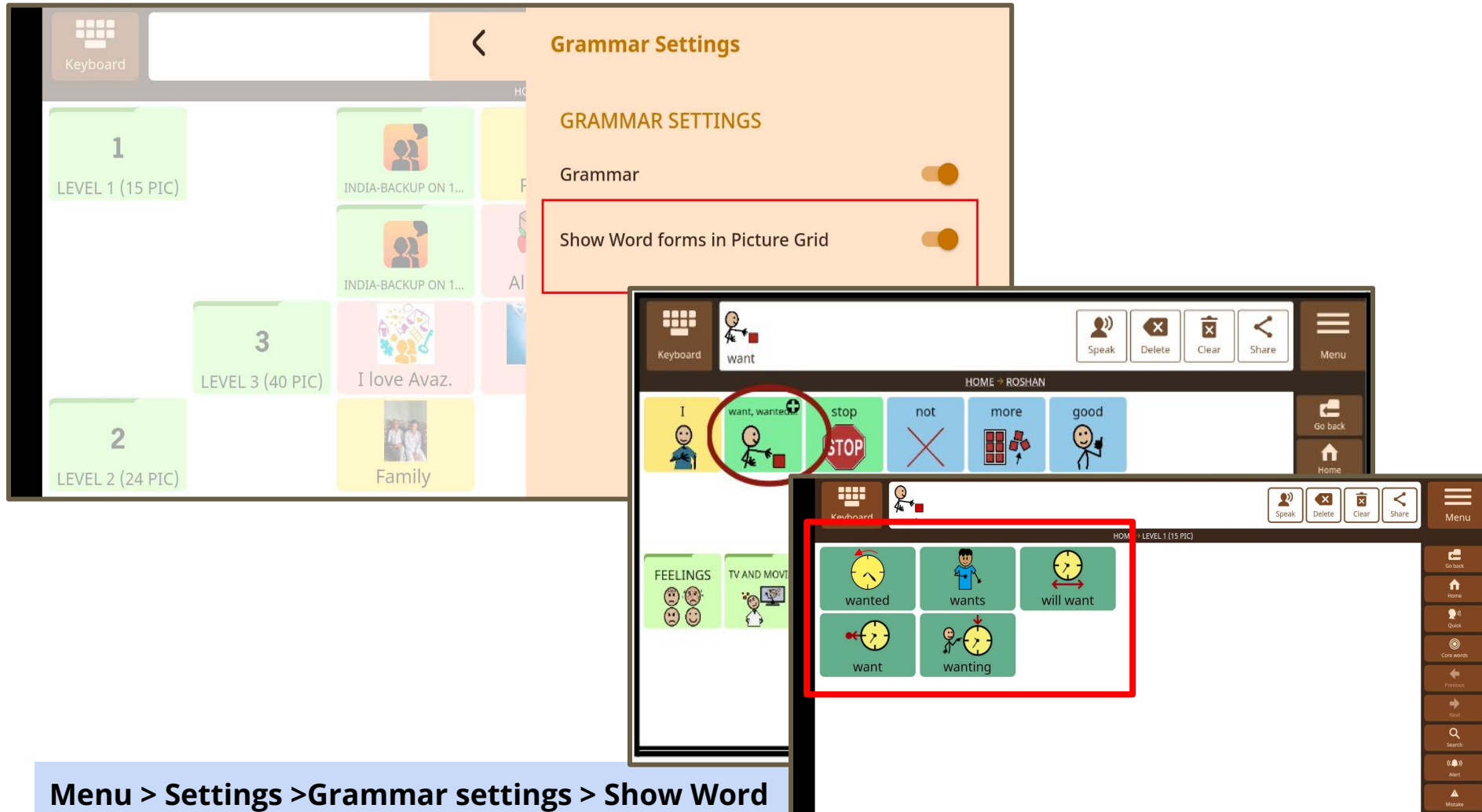

The image shows the Avaz AAC app interface. On the left, a grid of levels is visible: Level 1 (15 PIC), Level 2 (24 PIC), and Level 3 (40 PIC). The main screen displays the 'Grammar Settings' menu, which includes a toggle for 'Grammar' and a toggle for 'Show Word forms in Picture Grid'. The 'Show Word forms in Picture Grid' toggle is highlighted with a red box. Below the settings, a preview of the word form grid is shown. The grid displays the word 'want' in its base form, along with its morphological forms: 'wanted', 'wants', 'will want', 'wanting', and 'want'. The grid is organized into a 2x3 layout, with the word 'want' in the top-left cell, 'wanted' in the top-middle cell, 'wants' in the top-right cell, 'will want' in the bottom-left cell, 'wanting' in the bottom-middle cell, and an empty cell in the bottom-right. The grid is highlighted with a red box.

Grammar Settings

GRAMMAR SETTINGS

Grammar ☐


Show Word forms in Picture Grid ☒

Word Form Grid:

wanted	wants	will want
want	wanting	

Menu > Settings > Grammar settings > Show Word forms in Picture Grid


Backup & Restore - Share Backup

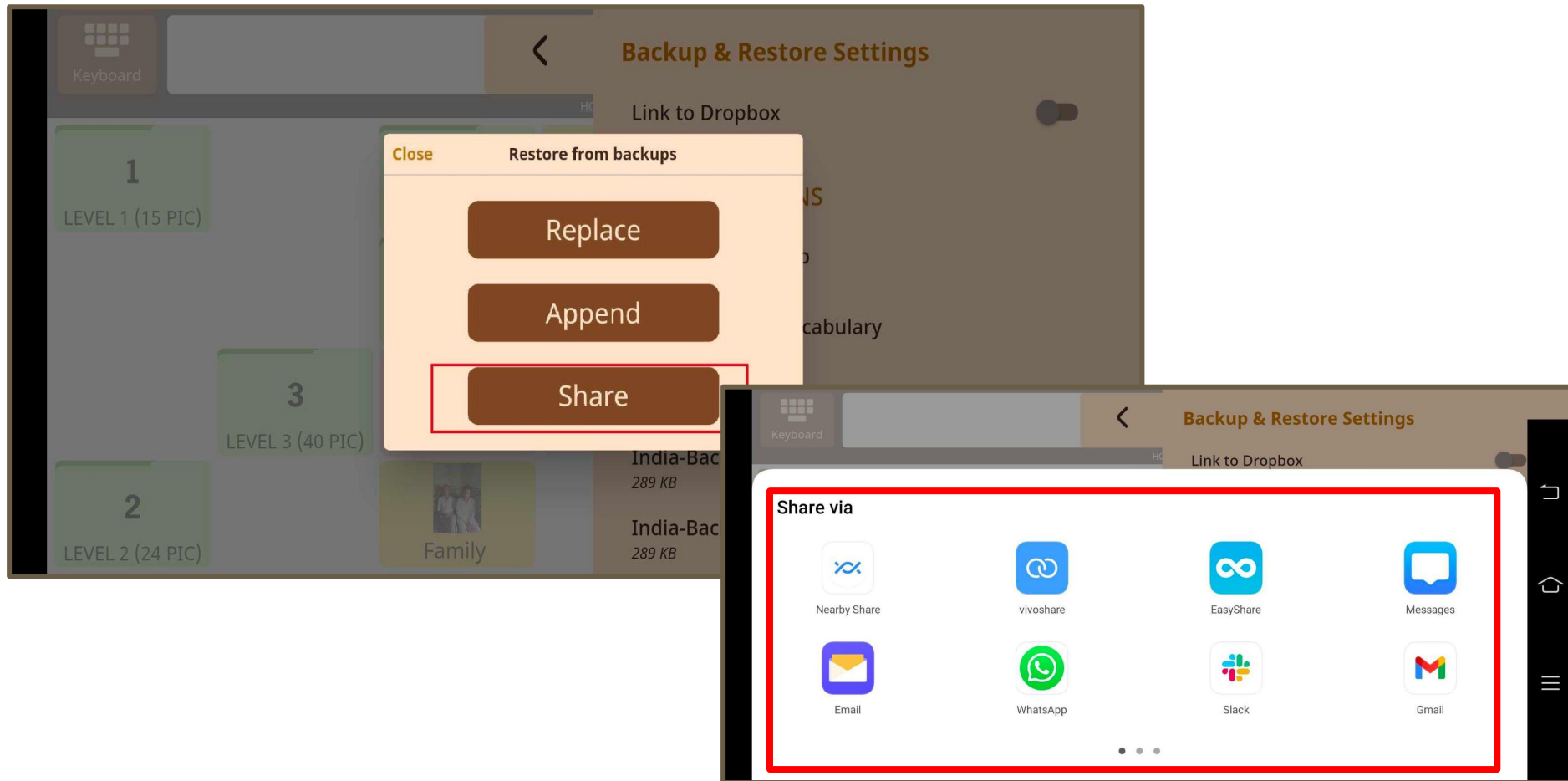

Menu > Settings > Backup & Restore > Select the Backup to be shared > Share


Select from the list of apps (to which the Backup can be shared) that are displayed.

Keyboard - Separate button for Action Keys

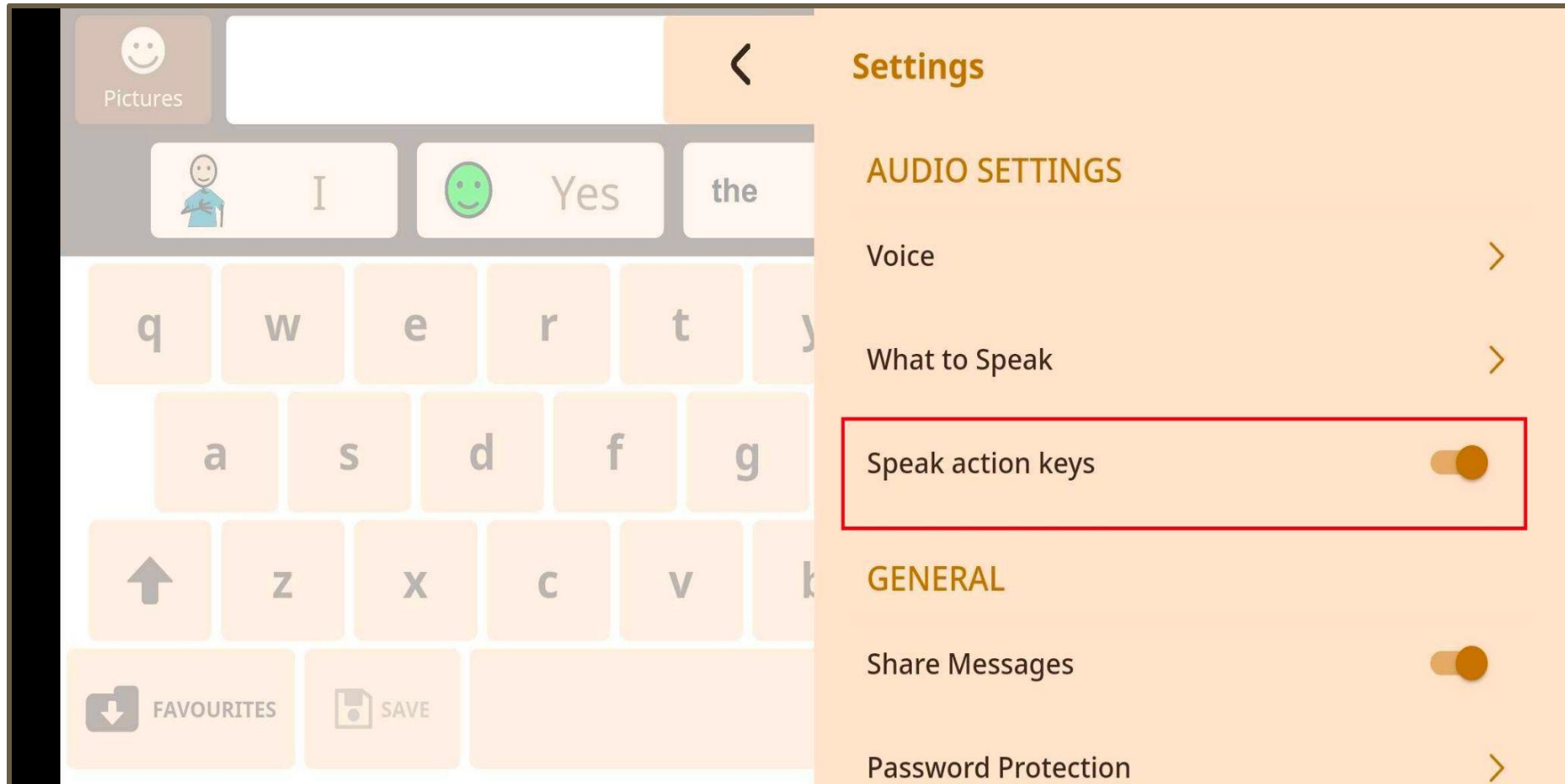


Menu > Settings > Audio Settings > Speak action keys

Keyboard - Favourites

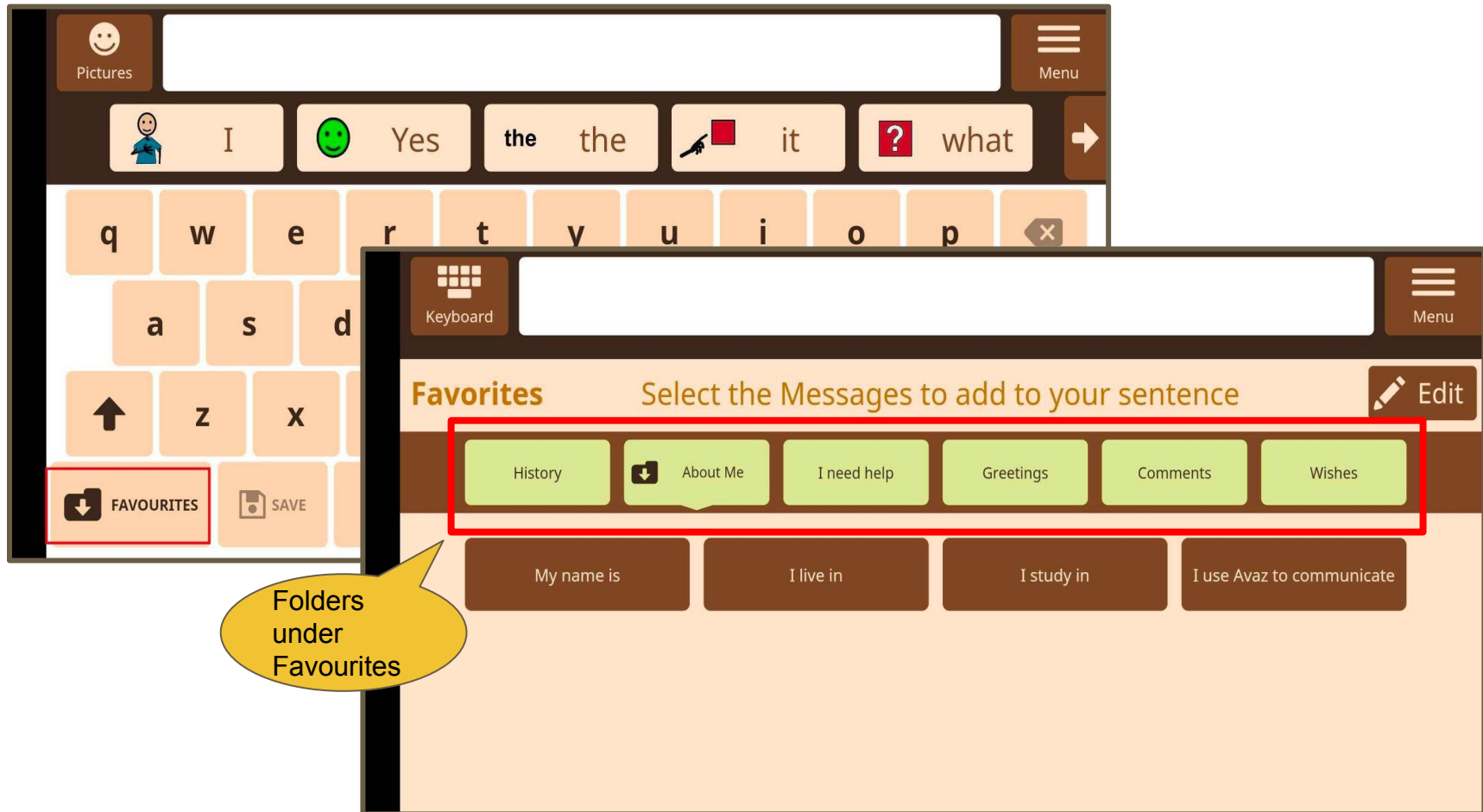

Favourites - to store frequently used phrases / sentences, organized under folders.


Keyboard - Favourites Folder

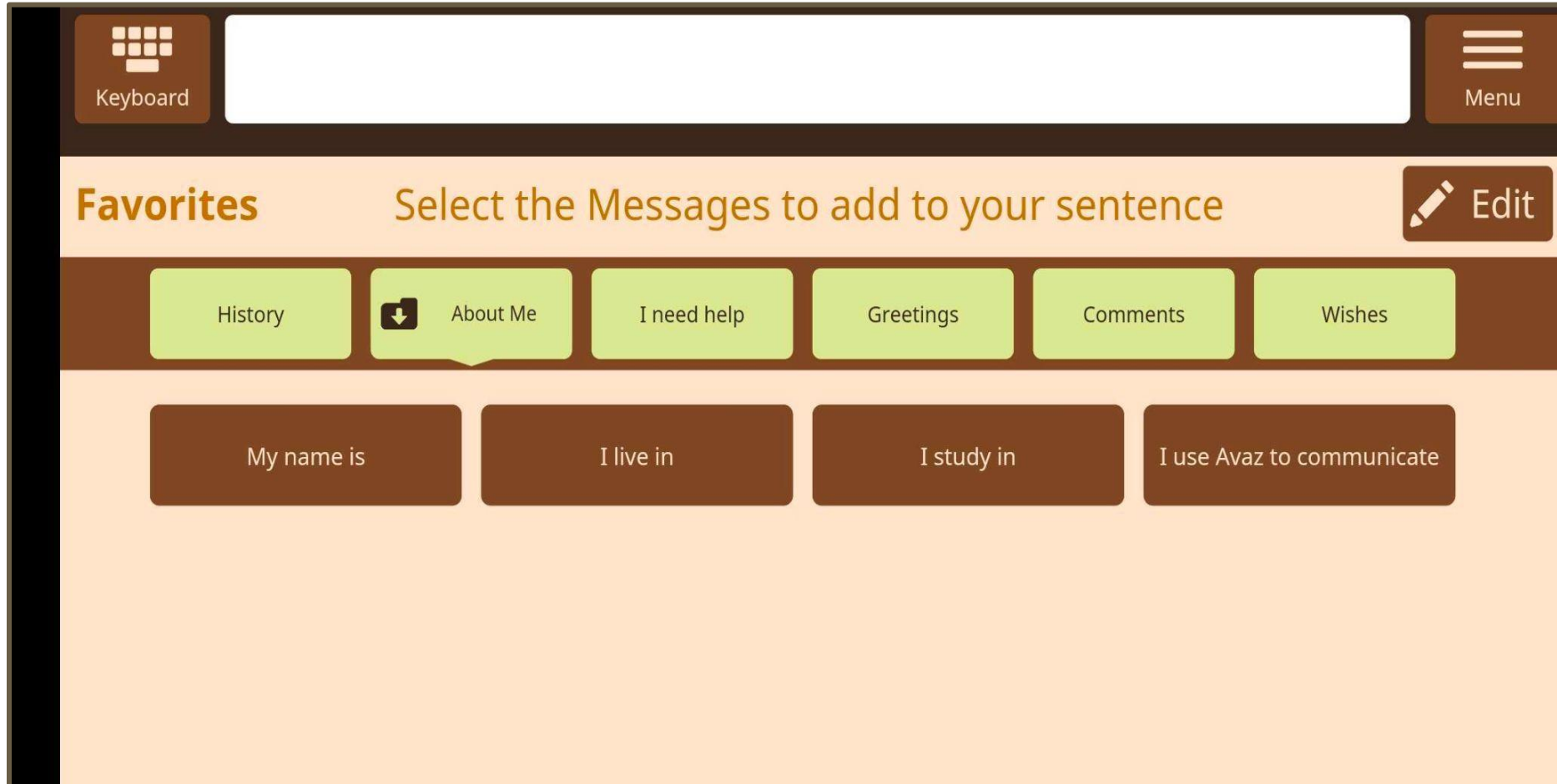

The screenshot displays the 'Keyboard - Favourites Folder' interface of the Avaz AAC app. At the top, there is a 'Keyboard' icon on the left and a 'Menu' icon on the right. Below this is a large white text input field. The main section is titled 'Favorites' and contains the instruction 'Select the Messages to add to your sentence'. To the right of this instruction is an 'Edit' button with a pencil icon. Below the instruction, there are six light green buttons: 'History', 'About Me' (which has a downward arrow icon), 'I need help', 'Greetings', 'Comments', and 'Wishes'. At the bottom, there are four dark brown buttons with white text: 'My name is', 'I live in', 'I study in', and 'I use Avaz to communicate'.



Under Favourites Folder, you may select the messages to add to your sentence

Keyboard - Edit Favourites Folder (Rename / Delete)

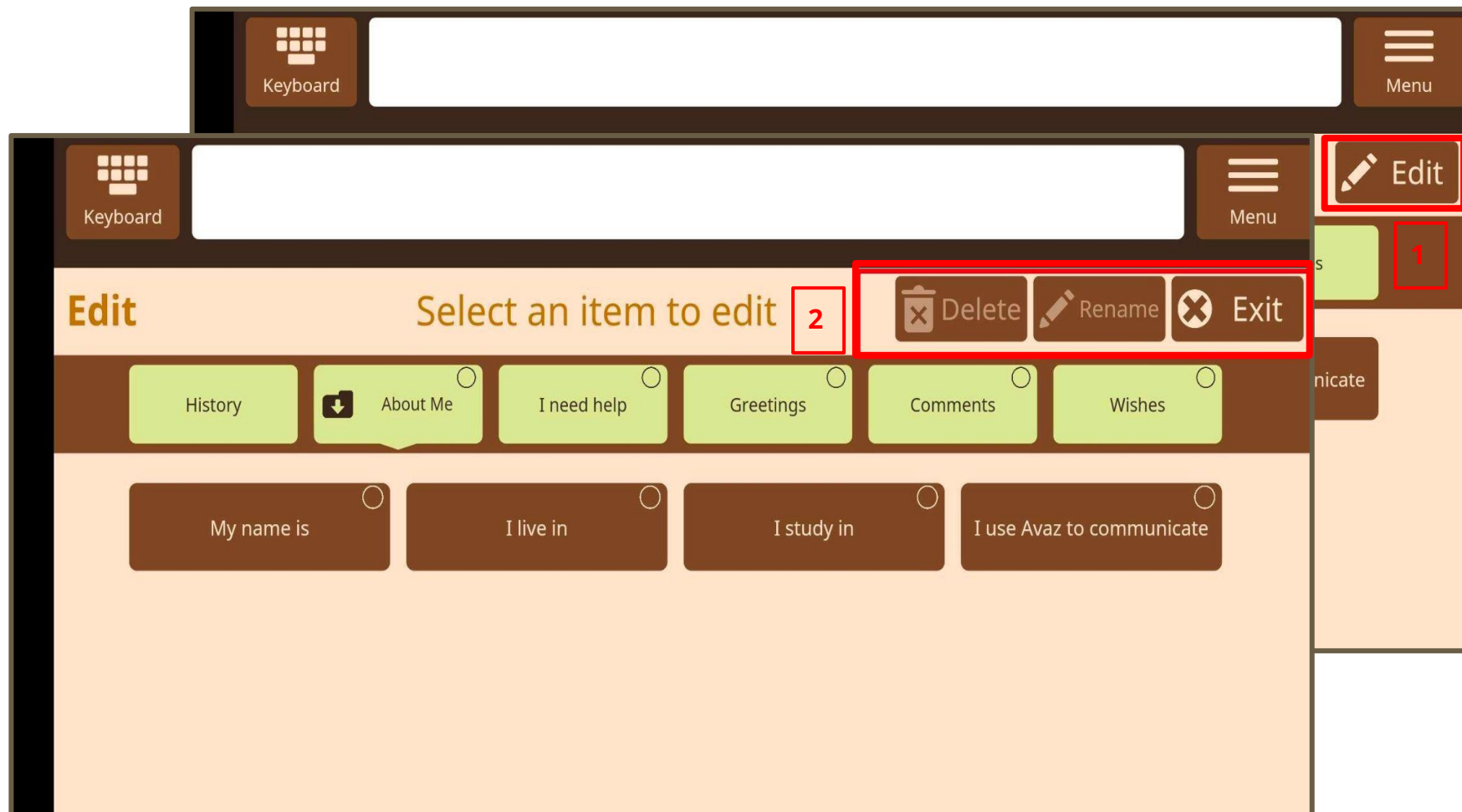


Favourites > Edit > Select a Folder / phrase to Delete or Rename.

Voices and Speech Settings

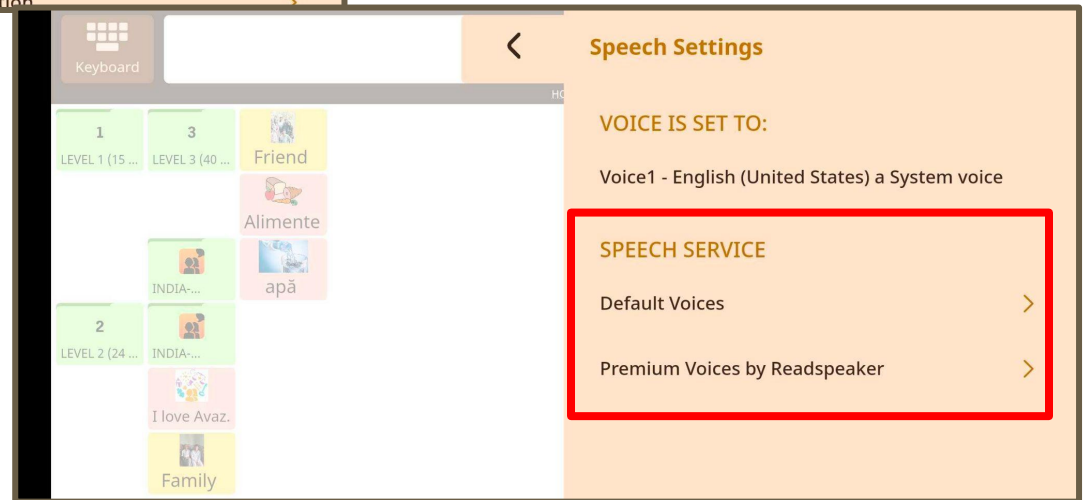
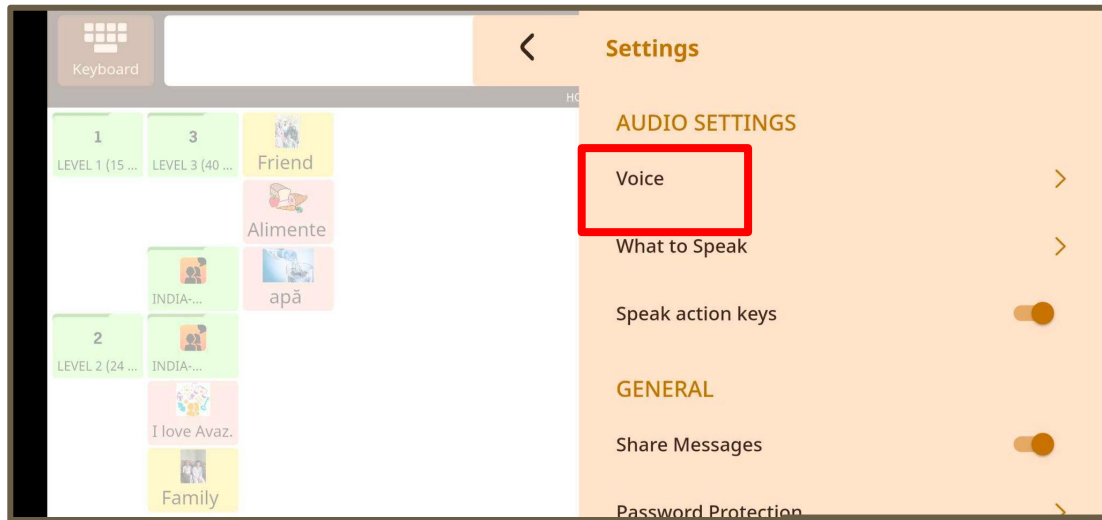


Menu>Settings>Audio>Voice

Voices and Speech Settings

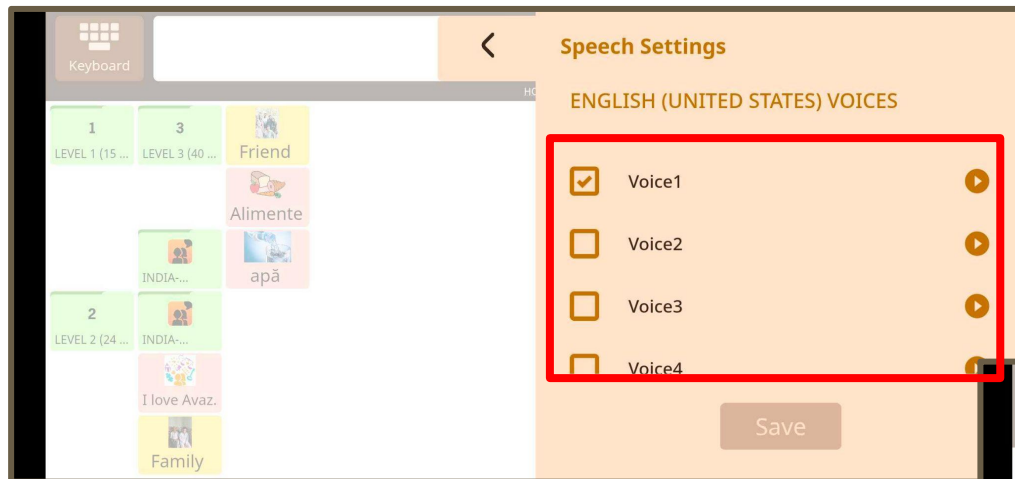




3. Voice > Default Voices>English Voices

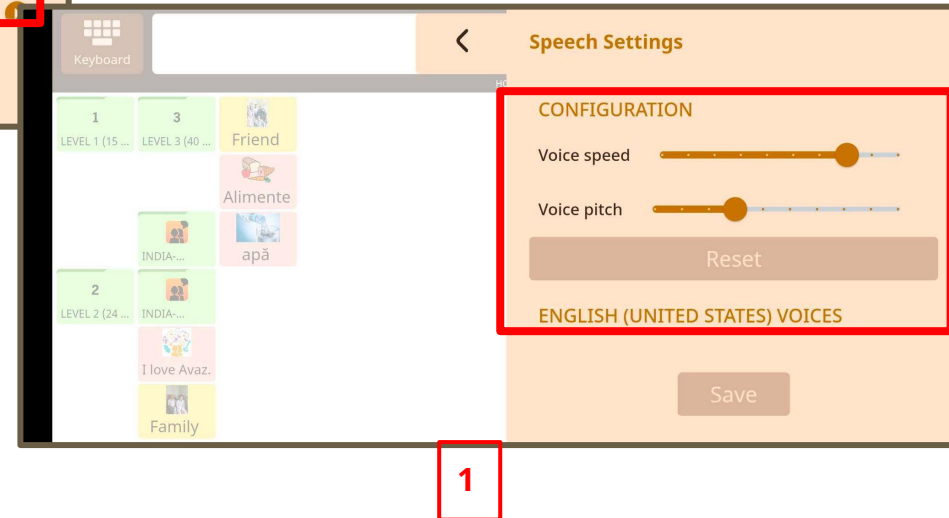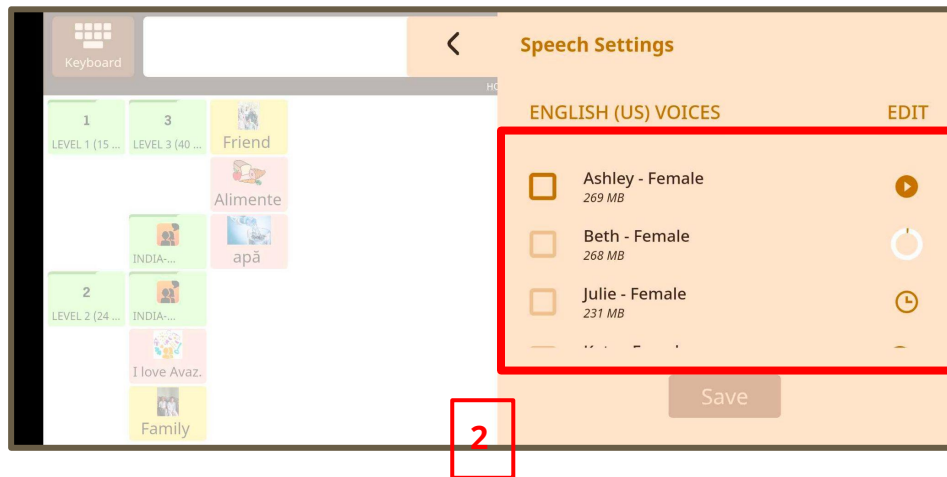

Note: Avaz has Default & Readspeaker voices

1. Voice > Default Voices> English (United States)/Other >Configuration (For Voice speed & pitch)
2. Voice > Default Voices> English (United States)/Other (For Voice types)

How do I Delete My Account?

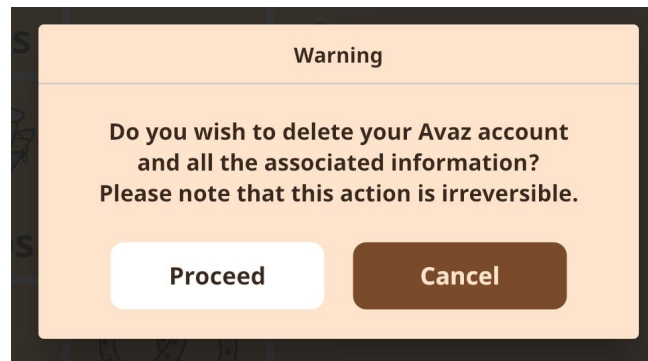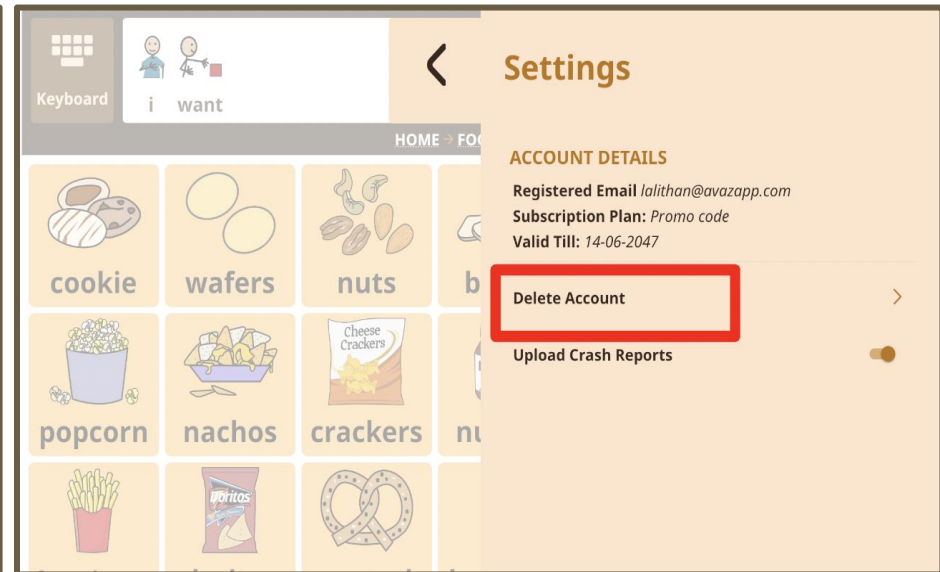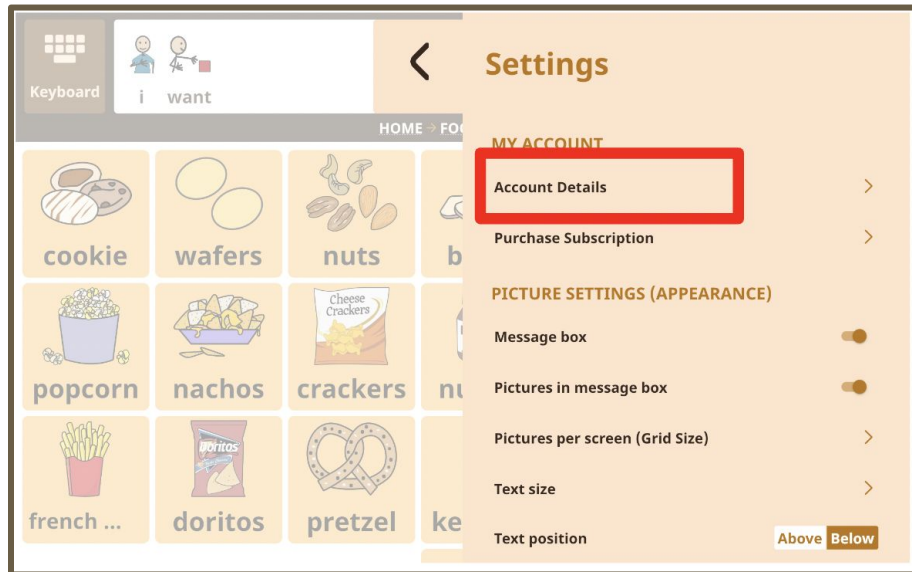


This option is available only from Avaz ver 6.6 and above and only for iOS apps. Note: This option is not available for Android apps. You can delete your account from the app. 1. Go to Settings > Account details. A window pops up for a confirmation of the Account deletion and all its associated information. Please note that the action is irreversible. Once your account is deleted it cannot be restored

Features removed

— From Avaz ver 6.6+ —



Buy Now button on home screen (removed)

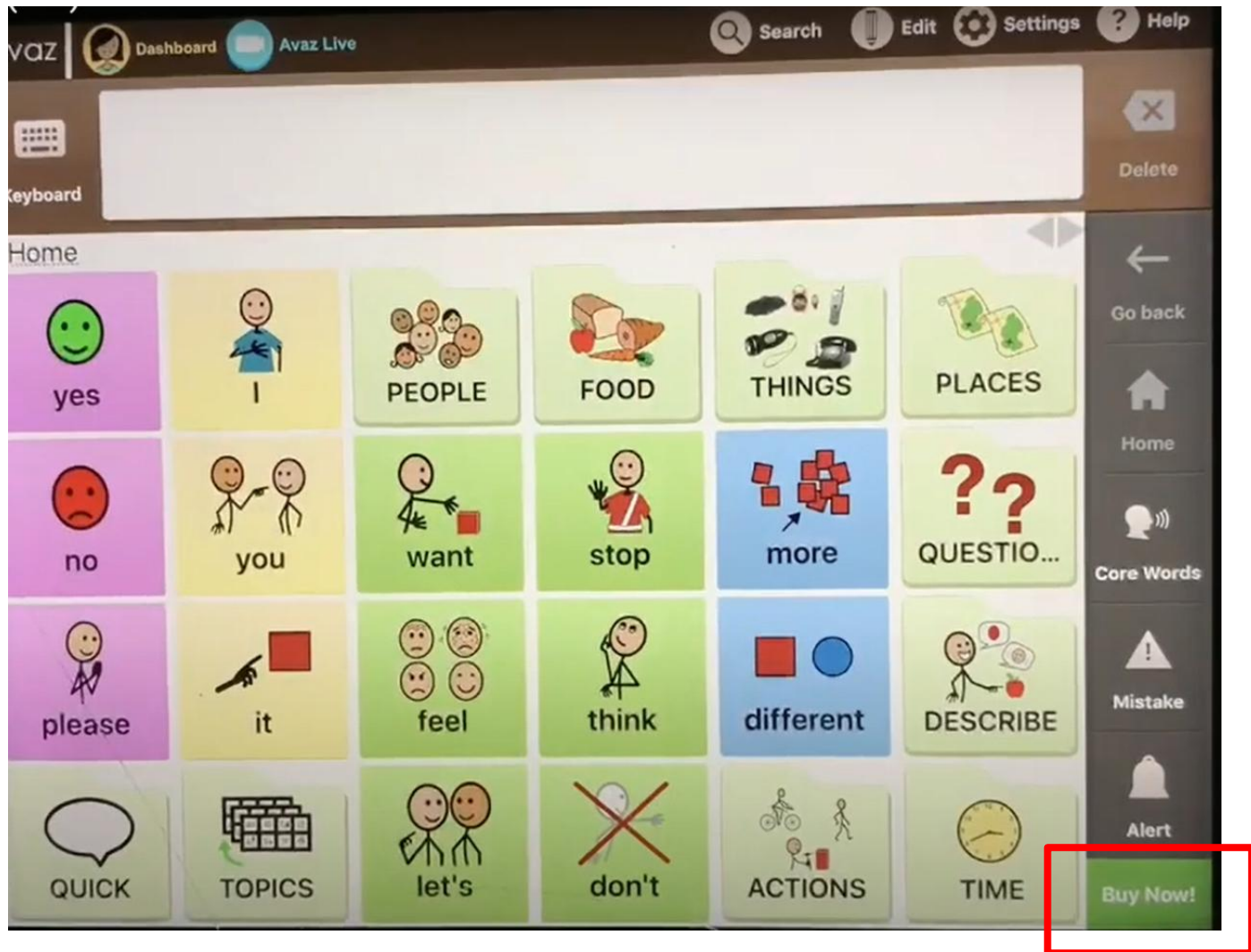

Dashboard (removed)

1

2

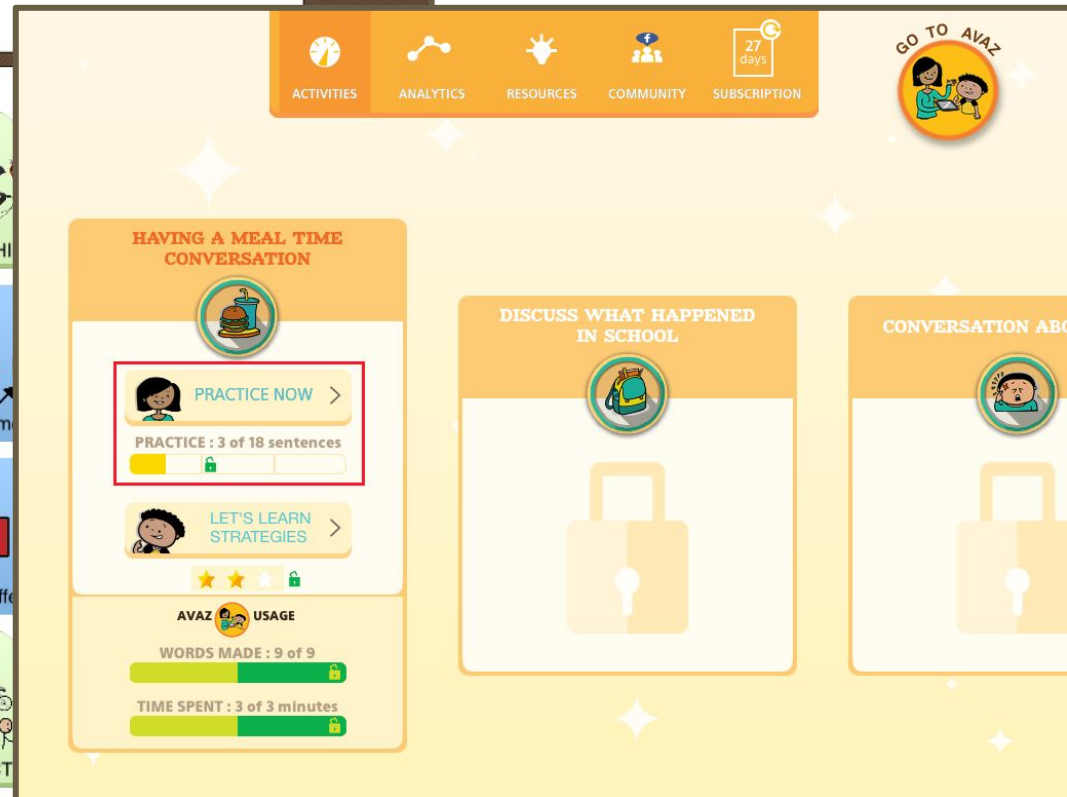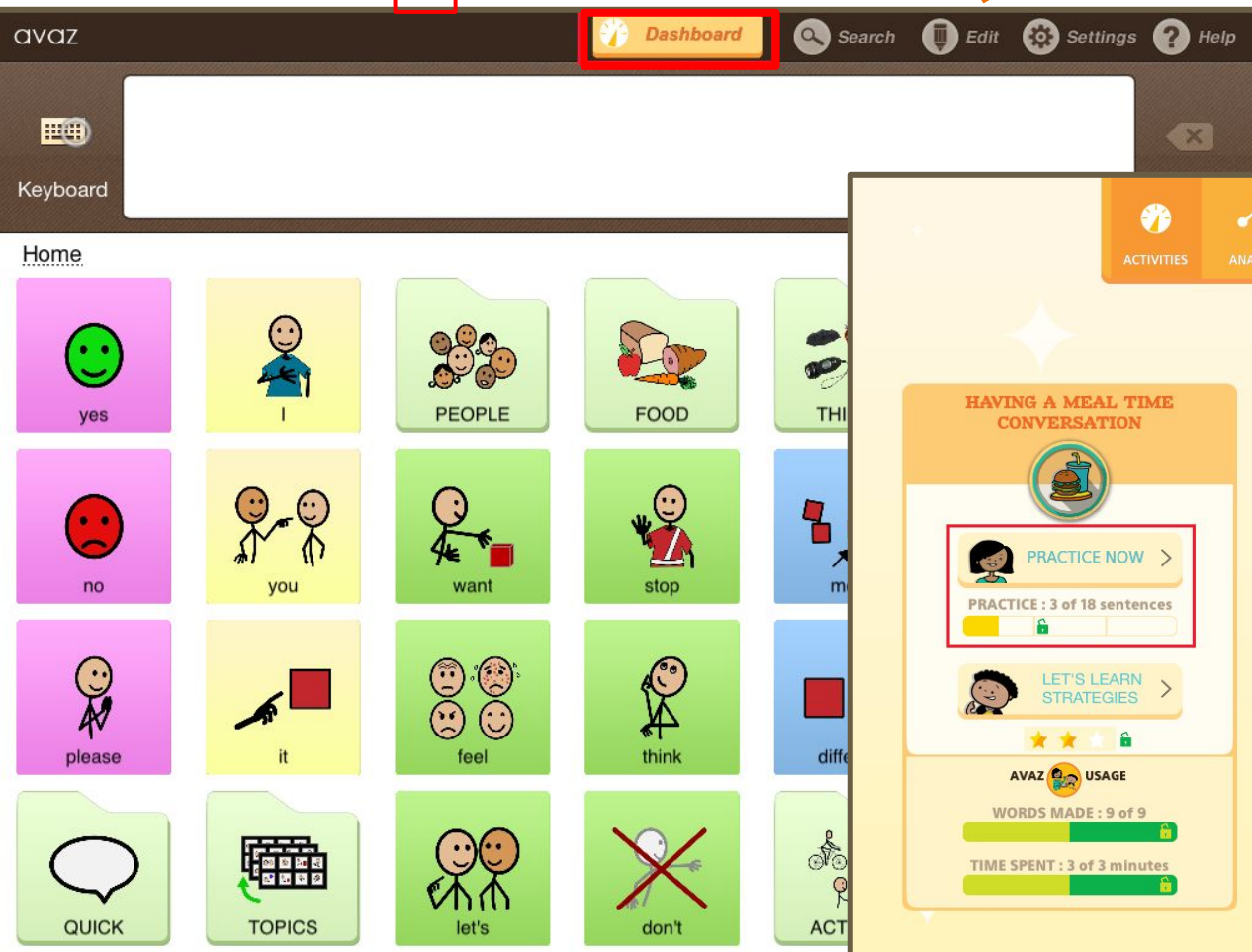

We are currently re-working the Dashboard feature. We plan to have a newer version of this much loved features soon, hence you will not be able to see this feature in the this current updated version (ver 6.6)

Avaz Live (Removed)



We are currently re-working the Avaz Live feature. We plan to have a newer version of this much loved feature soon, hence you will not be able to see this feature in the updated version.